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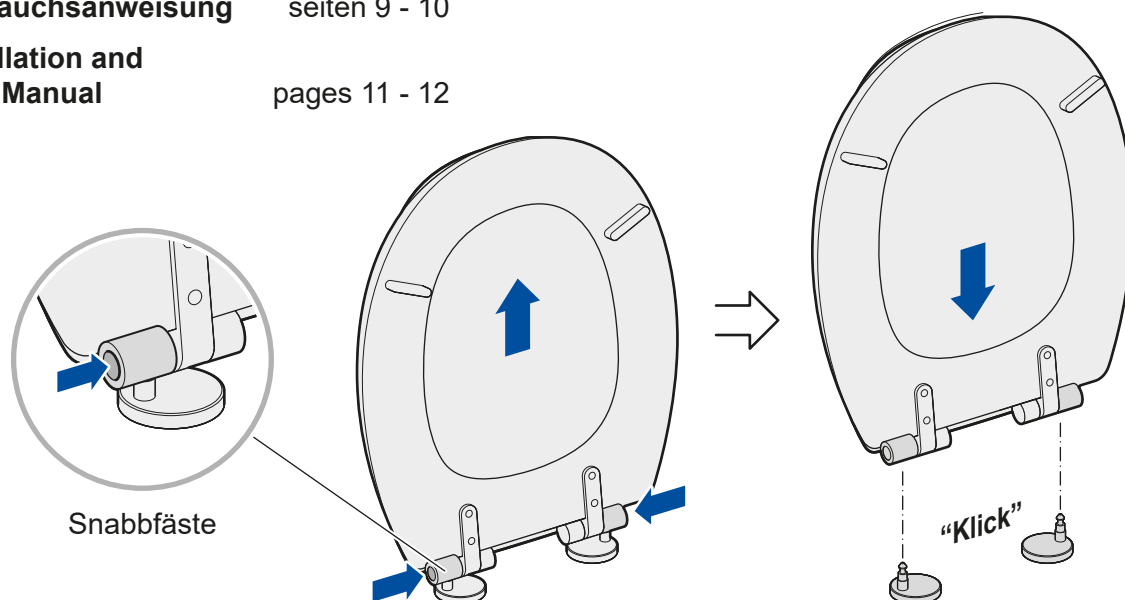
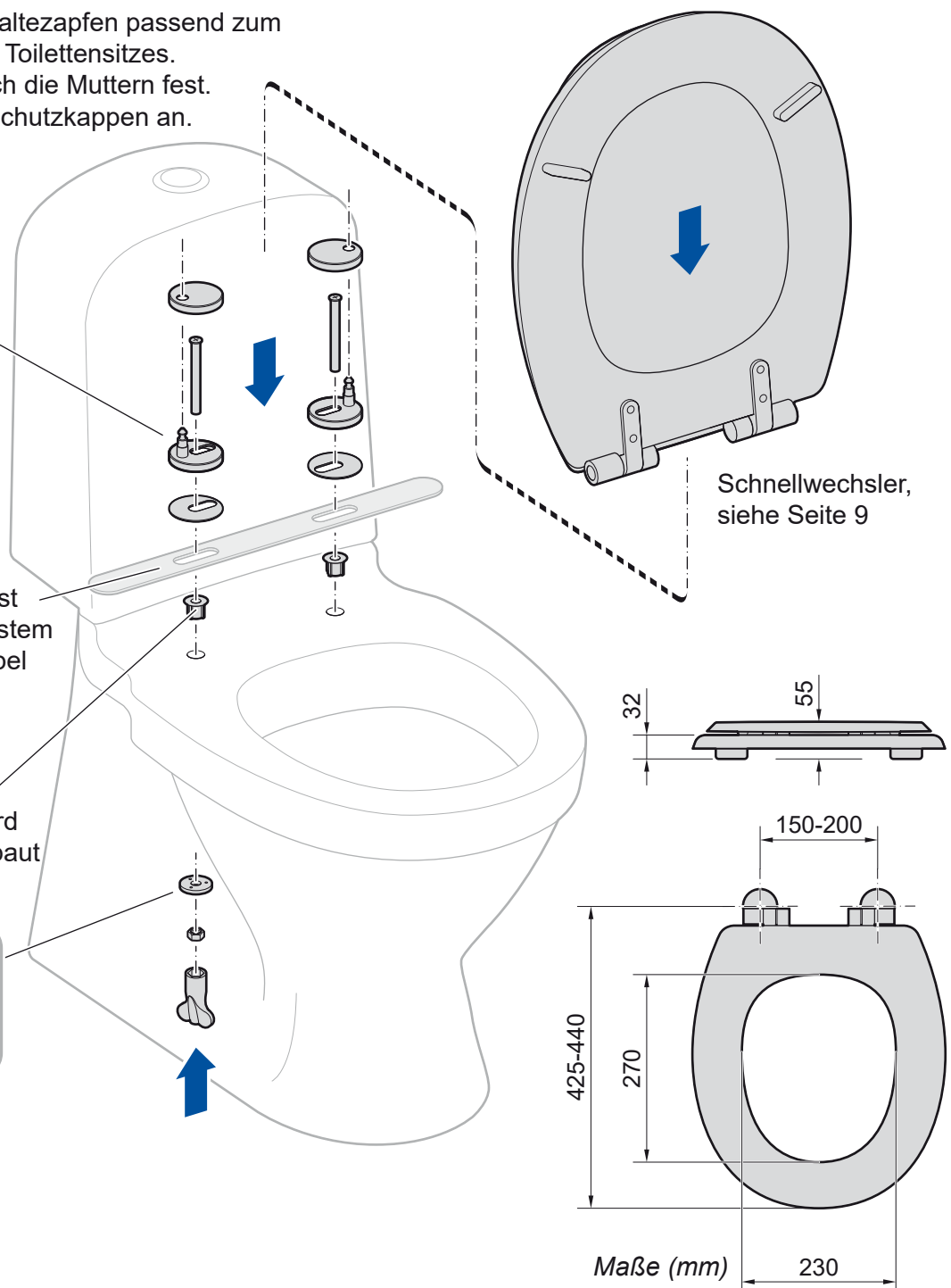
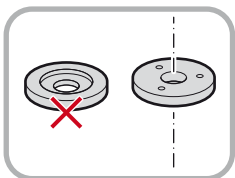
Toilet seat with soft closure

Item. no. 10 141


Read **Installation and User Manual** before mounting and using **Toilet Seat**. Save the manual for future reference.

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Declaration of Conformity

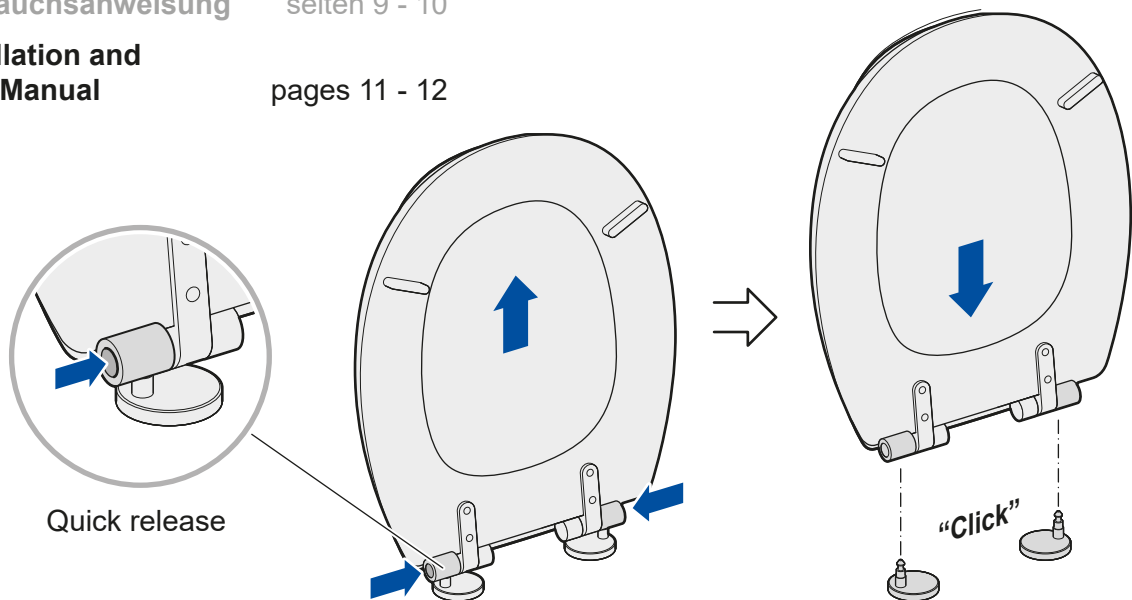This product is approved and CE marked according to regulation MDR 2017/745. See CE declaration on Svan Care's website.

Product description

The toilet seat has an oil-damped soft closure on the lid and seat. The fittings have a quick-release bracket, which means that the toilet seat can be detached and mounted with a simple touch (facilitates cleaning, for example).

The toilet seat can be combined with Balance (then replaces Standard seat, Item. no. 10041).

The toilet seat can also be combined with:
Hole reducer 10046, 10047, 10049, and 10050.
Soft seats 10398 and 10399.

Cleaning

If necessary, clean the product with a mild detergent and water. Disinfect if necessary.

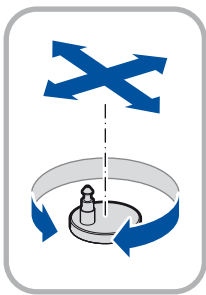
Reconditioning

Disassemble the entire product. The parts are washed clean with non-scratching detergent, rinsed with water, and dried. Disinfect. Check all parts and replace in case of wear.

Materials, environment, and recycling

Toilet seat and lid are made of recycled wood fibre with a coated polyester coating. Metal fittings. Toilet seat and lid are sent for incineration or recycling. Metal parts are recycled as metal.

Turn the pin bracket to fit the hole distance of the toilet seat.
Then tighten the nuts.
Fit the protective caps.

The toilet seat also fits the Balance module system

Bushing for stabilisation, used if necessary