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
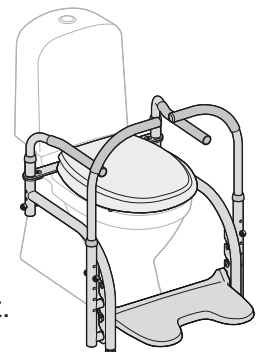
Read **Installation and User Manual** before install and use of **Balance**. Save manual for future use.

Balance

Item number 10 101 - 10 110

*The User Manual is applicable from batch number 214544 onwards.
These products include the soft-closing seat.*

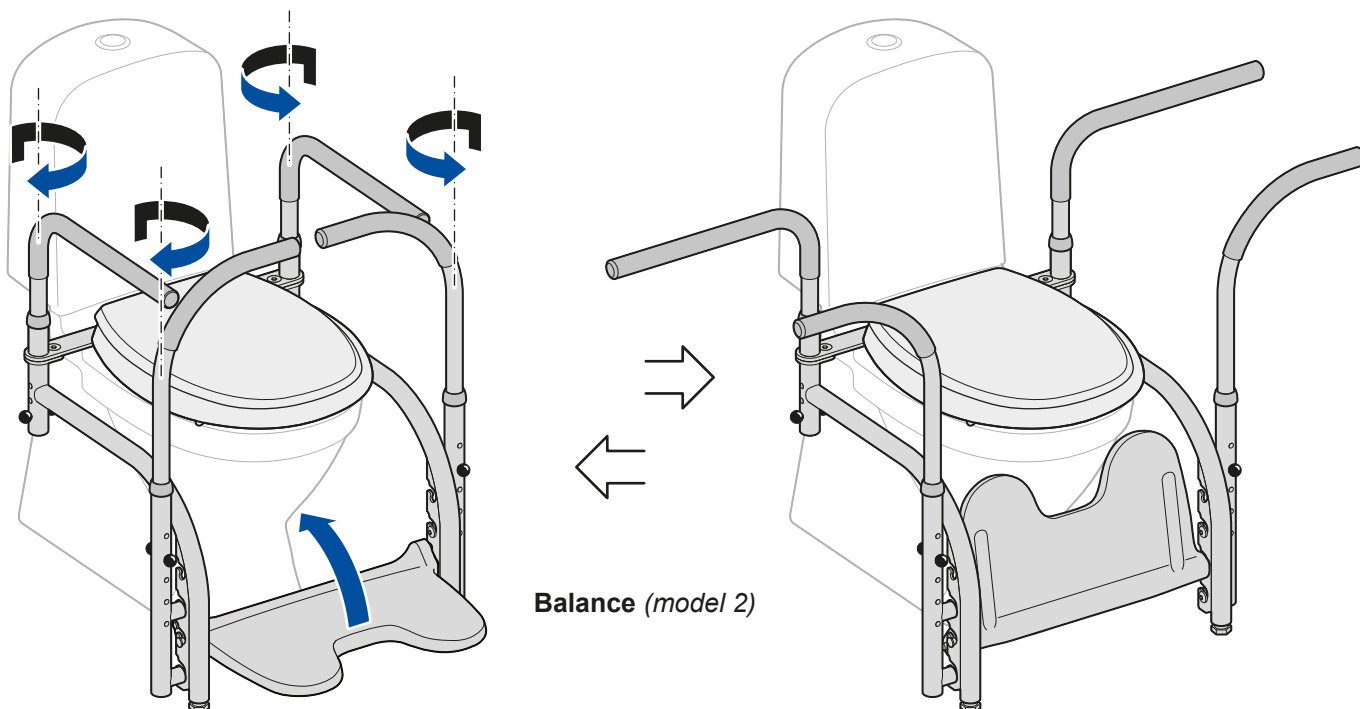Product description

Balance is an aid for those needing extra support when moving to and from the toilet and when sitting on the toilet.

Balance is a modular system consisting of many different components that can be combined in a variety of ways.

The toilet seat is equipped with an oil-damped soft-close lid and seat.

The fittings have a quick attachment facility, meaning that the toilet seat can easily be detached and re-attached (making cleaning simpler, for example – see page 35).

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
Seiten 25 - 30

Installation, User Manual and Reconditioning

pages 31 - 36

Exploded view

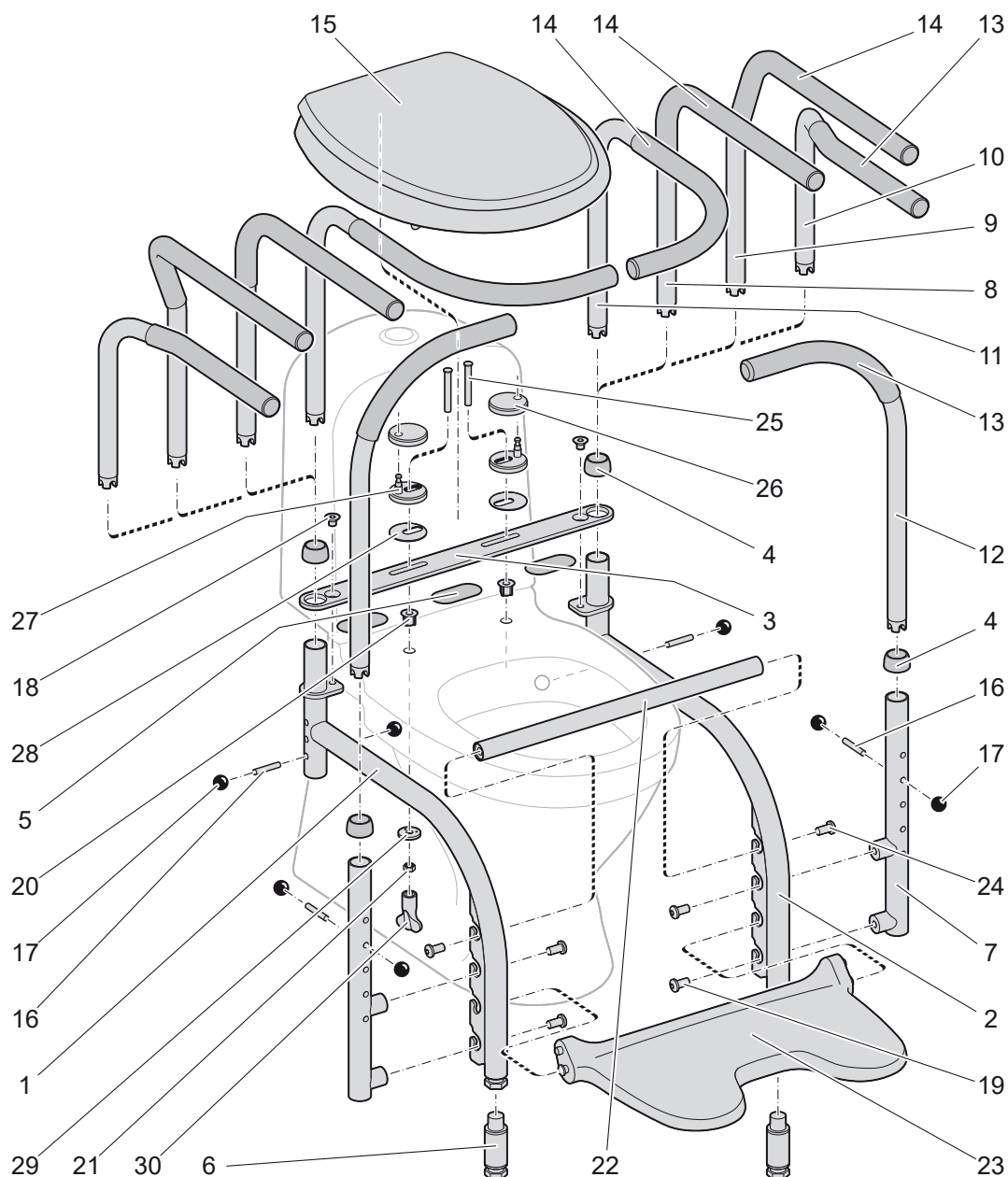Svan Care reserves the right to make changes to the parts list. Current information is available on our website.

Parts list

No	Part	Item number
1	Right support leg (included in support leg, part no. 10 002)	Included in 10 002
2	Left support leg	Included in 10 002
3	Transverse plate	Included in 10 002
4	Plastic cover (included in support leg and attachment tube for support handle)	Included in 10 002
5	Rubber insert (36 x 73 mm), piece	10 077
6	Support leg extenders, pair (used for raised toilet bowls, 45 cm or higher)	10 087
7	Attachment tubes for support handle, piece (pos. no. 4, 16, 17 included)	10 031
8	Armrest Straight, pair	10 052
9	Armrest Duo, pair	10 053
10	Armrest Duo Mini, pair	10 084
11	Armrest Surround, pair	10 120
12	Support handle, piece	10 091
13	Rubber handle (270 mm)	10 056
14	Rubber handle (440 mm)	10 055
15	Toilet seat, soft-closing with quick attachment	10 141

No	Part	Item number
16	Locking pin	10 008
17	Plastic ball	10 009
18	Countersunk screw	Included in 10 002
19	Screw (included in attachment tube for support handle)	Included in 10 031
20	Bushes for stabilisation (included in toilet seat mounting kit)	Included in 10 142
21	Nut (included in toilet seat mounting kit)	Included in 10 142
22	Footrest replacement	10 038
23	Footrest	10 039
24	Screw (included in footrest replacement)	Included in 10 038
25	Screw (included in toilet seat mounting kit)	Included in 10 142
26	Protective cover (included in toilet seat mounting kit)	Included in 10 142
27	Pin bracket (included in toilet seat mounting kit)	Included in 10 142
28	Gasket (included in toilet seat mounting kit)	Included in 10 142
29	Washer (included in toilet seat mounting kit)	Included in 10 142
30	Spanner (included in toilet seat mounting kit)	Included in 10 142

Use

Check that **Balance** is firmly attached to the toilet and that it is installed correctly before use.

Cleaning

As required, clean the product with a mild detergent and water. Disinfect where necessary.

Reconditioning

Disassemble removable parts. Wash the parts clean (at a maximum of 80°C for 30 min) with a non-scratching detergent, rinse with water and dry. Disinfect. Check all parts and replace if worn.

Materials, environment and recycling

Balance is constructed from lacquered stainless steel and its details from polyurethane without latex. The toilet seat is made of recycled wood fibre with a coating of polyester that is lacquered (without latex). Its fittings are of metal. When they are to be disposed of, the toilet seat, footrest and rubber cover are sent for incineration or recycling. The metal parts are recycled as metal.

Specifications

Balance can be combined with several of Svan Care's products, e.g. Bidette, Hole Reducer, Back Cushion etc. These are accompanied by their own user manuals.

Important!

Balance should only be installed on toilets that are well anchored to the floor or wall.

Never leave a person using **Balance** unattended.

Use only original Svan Care accessories. Svan Care assumes no responsibility for the product if it is used, altered or installed in any way other

than as specified in this user manual.

Maximum loads:

75 kg/Armrest (150 kg/pair),

75 kg/Armrest Surround (150 kg/pair),


75 kg/Support handle (150 kg/pair),

Support leg 150 kg, Footrest 60 kg

Estimated technical life: 5 years

Mounting of support legs, toilet seat, support handles and armrests

ATTENTION! Check which model you have (not all parts come with all models).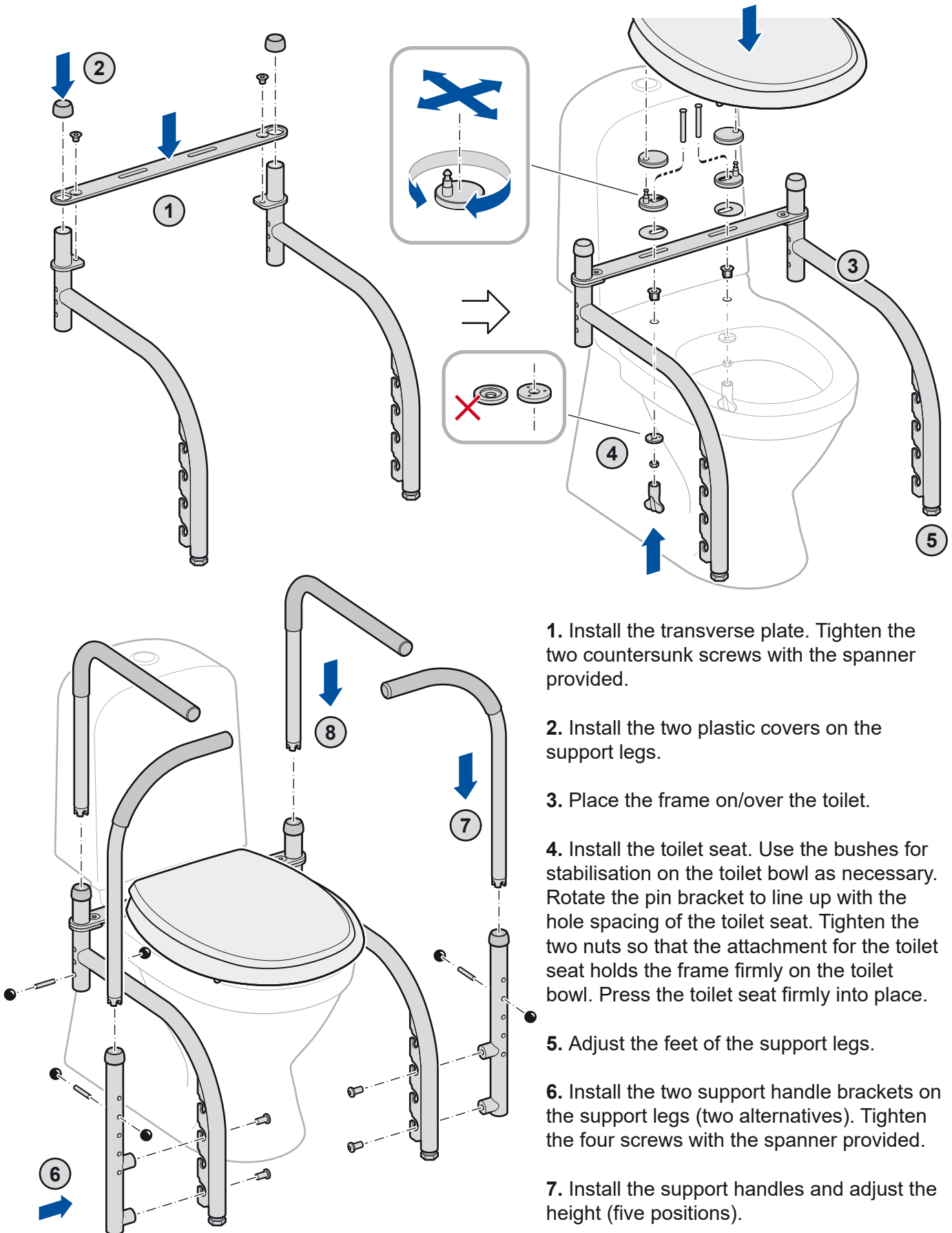

1. Install the transverse plate. Tighten the two countersunk screws with the spanner provided.

2. Install the two plastic covers on the support legs.

3. Place the frame on/over the toilet.

4. Install the toilet seat. Use the bushes for stabilisation on the toilet bowl as necessary. Rotate the pin bracket to line up with the hole spacing of the toilet seat. Tighten the two nuts so that the attachment for the toilet seat holds the frame firmly on the toilet bowl. Press the toilet seat firmly into place.

5. Adjust the feet of the support legs.


6. Install the two support handle brackets on the support legs (two alternatives). Tighten the four screws with the spanner provided.

7. Install the support handles and adjust the height (five positions).

8. Install the armrests and adjust the height (three positions).

Note. The locking pin plastic balls sometimes need to be tightened afterwards.



Mounting of footrest (or footrest replacement)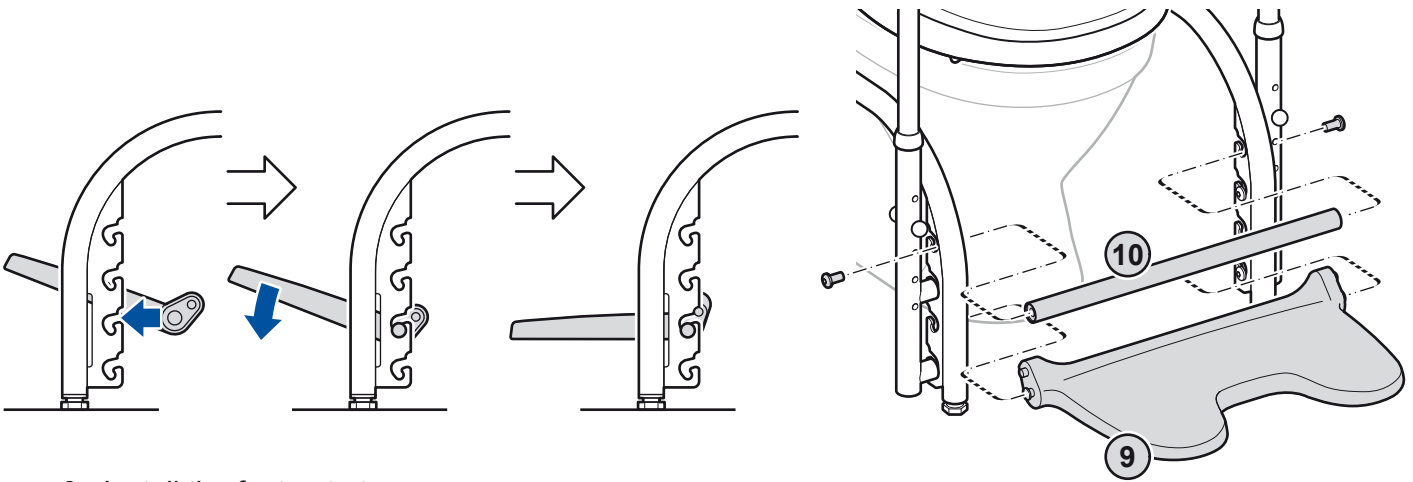

9. Install the footrest at an appropriate height.


10. If the footrest is not to be used, install the footrest replacement in the upper position. Tighten the two screws with the spanner provided.

11. Soft-closing toilet seat with quick attachment.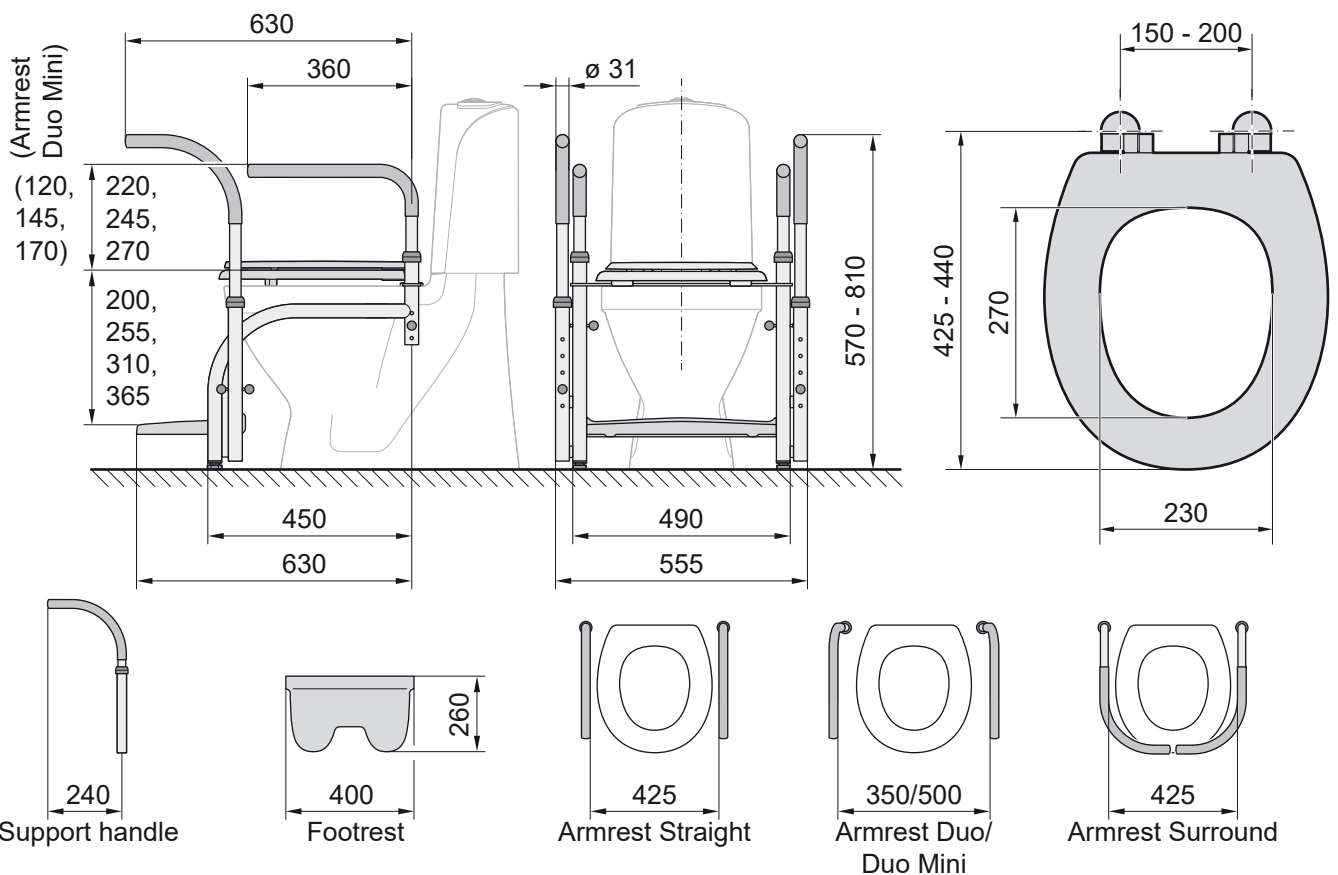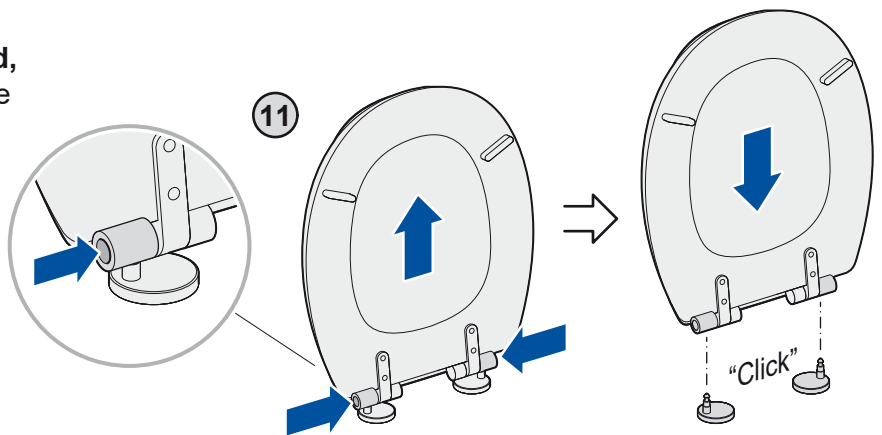


Models of Balance


Model 1: Item no. 10 101
 Armrest Straight, 425 mm
 between armrests.
 Footrest.
 Frame and toilet seat.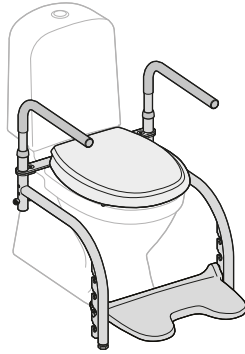


Model 2: Item no. 10 102
 Armrest Straight, 425 mm
 between armrests.
 Footrest.
 Support handles.
 Frame and toilet seat.

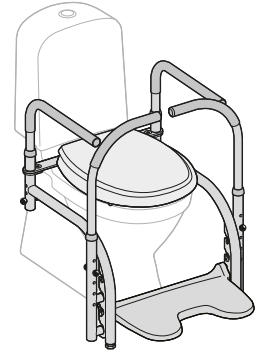
Model 3: Item no. 10 103
 Armrest Straight, 425 mm
 between armrests.
 Support handles.
 Frame and toilet seat.
 Footrest replacement.

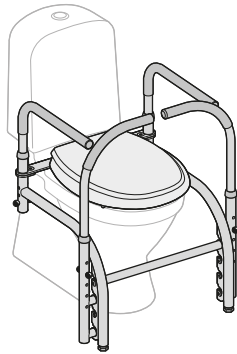
Model 4: Item no. 10 104
 Armrest Duo, 350 or
 500 mm between armrests.
 Footrest.
 Frame and toilet seat.

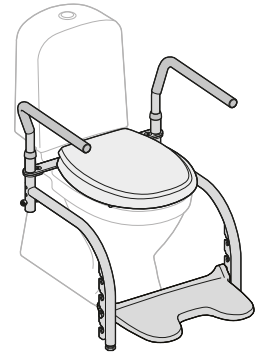
Model 5: Item no. 10 105
 Armrest Duo, 350 or
 500 mm between armrests.
 Footrest.
 Support handles.
 Frame and toilet seat.

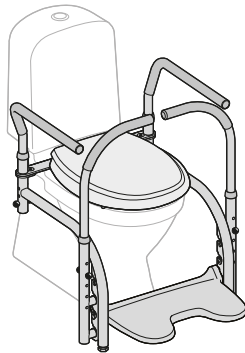
Model 6: Item no. 10 106
 Armrest Duo, 350 or
 500 mm between armrests.
 Support handles.
 Frame and toilet seat.
 Footrest replacement.

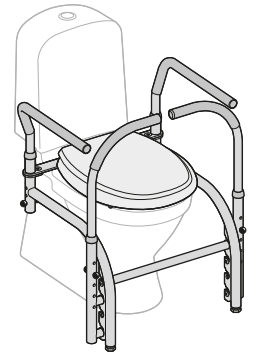
Model 7: Item no. 10 107
 Armrest Surround, 425 mm
 between armrests.
 Footrest.
 Frame and toilet seat.

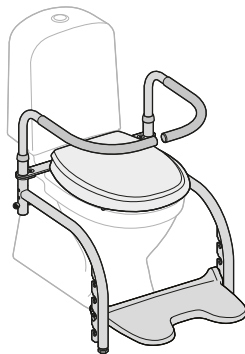Model 8: Item no. 10 108
 Armrest Surround, 425 mm
 between armrests.
 Frame and toilet seat.
 Footrest replacement.

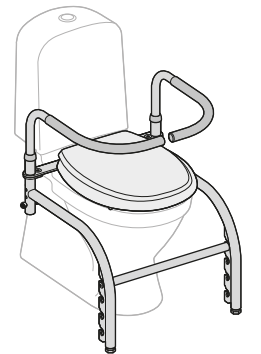Model 9: Item no. 10 109
 Armrest Duo Mini, 350 or
 500 mm between armrests.
 Footrest.
 Frame and toilet seat.

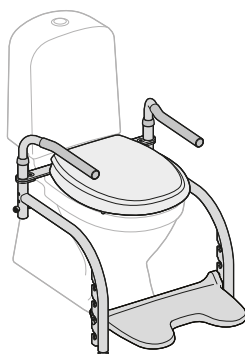Model 10: Item no. 10 110
 Armrest Duo Mini, 350 or
 500 mm between armrests.
 Footrest.
 Support handles.
 Frame and toilet seat.