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Art. nr. 10 415

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
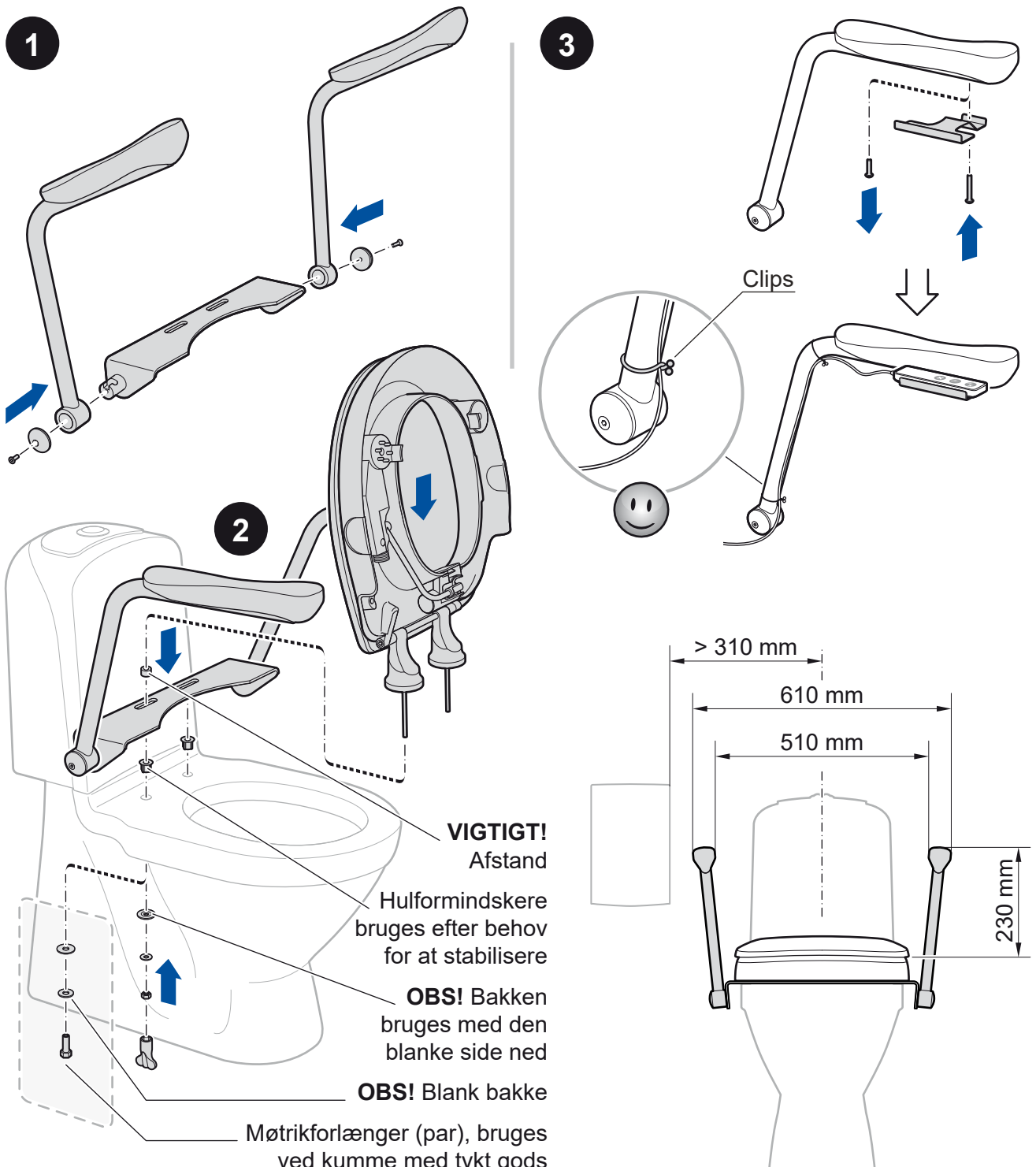
Grip armrest for Bidette

Item no. 10 415

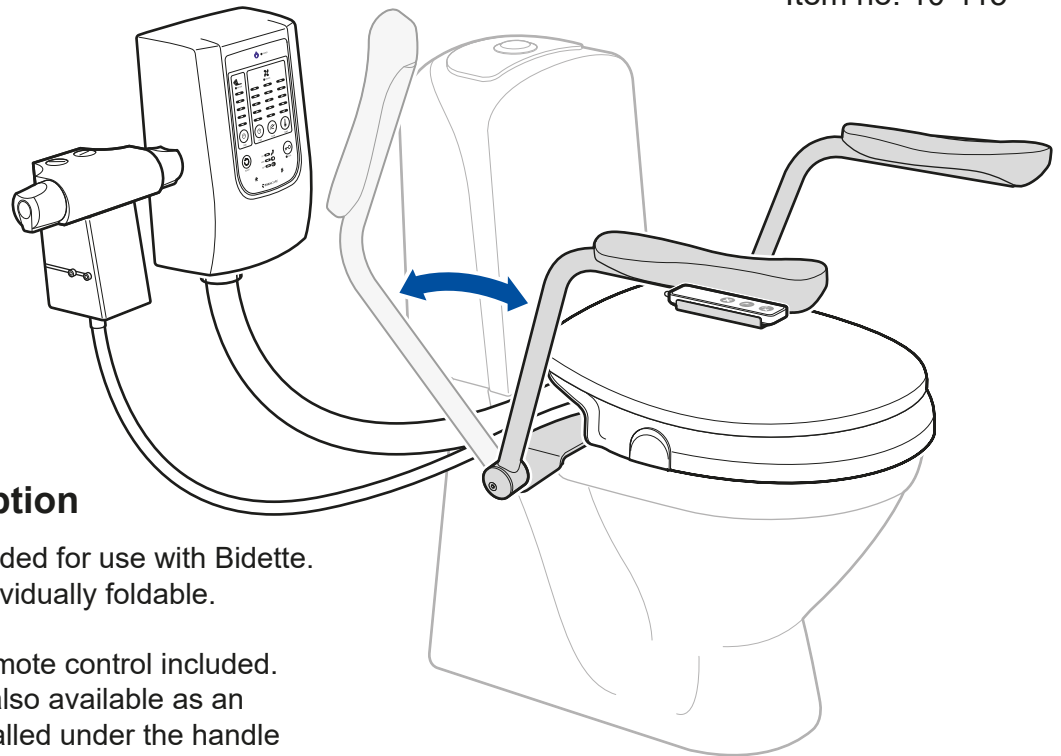Read **Installation and User Manual** prior to the installation and use of **Grip armrest**. Retain the manuals for future reference.

Product Description

Grip armrest is intended for use with Bidette. The armrests are individually foldable.

Mounting plate for remote control included. A toilet roll holder is also available as an accessory, to be installed under the handle part. Item no. 11495.

Grip armrest (installed on Bidette)

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sidorna 1 - 2

Montering og brugsanvisning

siderne 5 - 6

Monterings- og bruksanvisning

side 3 - 4

Installation and User Manual

pages 7 - 8

Important!

Grip armrest may only be installed on toilets that are well anchored to the floor or wall.

Use only original accessories from Svan Care. Svan Care assumes no responsibility if the product is used, changed or installed in any manner other than that given in these instructions.

Max load: 75 kg/armrest

Estimated technical lifetime: 5 years

Cleaning

As needed, clean the product with soap and water. Use 70% rubbing alcohol if necessary.

Reconditioning

Disassemble the entire product. The parts are washed clean with detergent, rinsed with water and dried. Disinfect. Check all parts for wear and tear, and replace if necessary.

Materials, environment and refuse sorting

Grip armrest is made of lacquered stainless steel with a latex-free rubber grip. The grips are sent off for incineration or recycling. Metal parts are recycled as metal.