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Armstöd Bidette för Bidette R3, M3, P12

Art. nr. 11 401

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
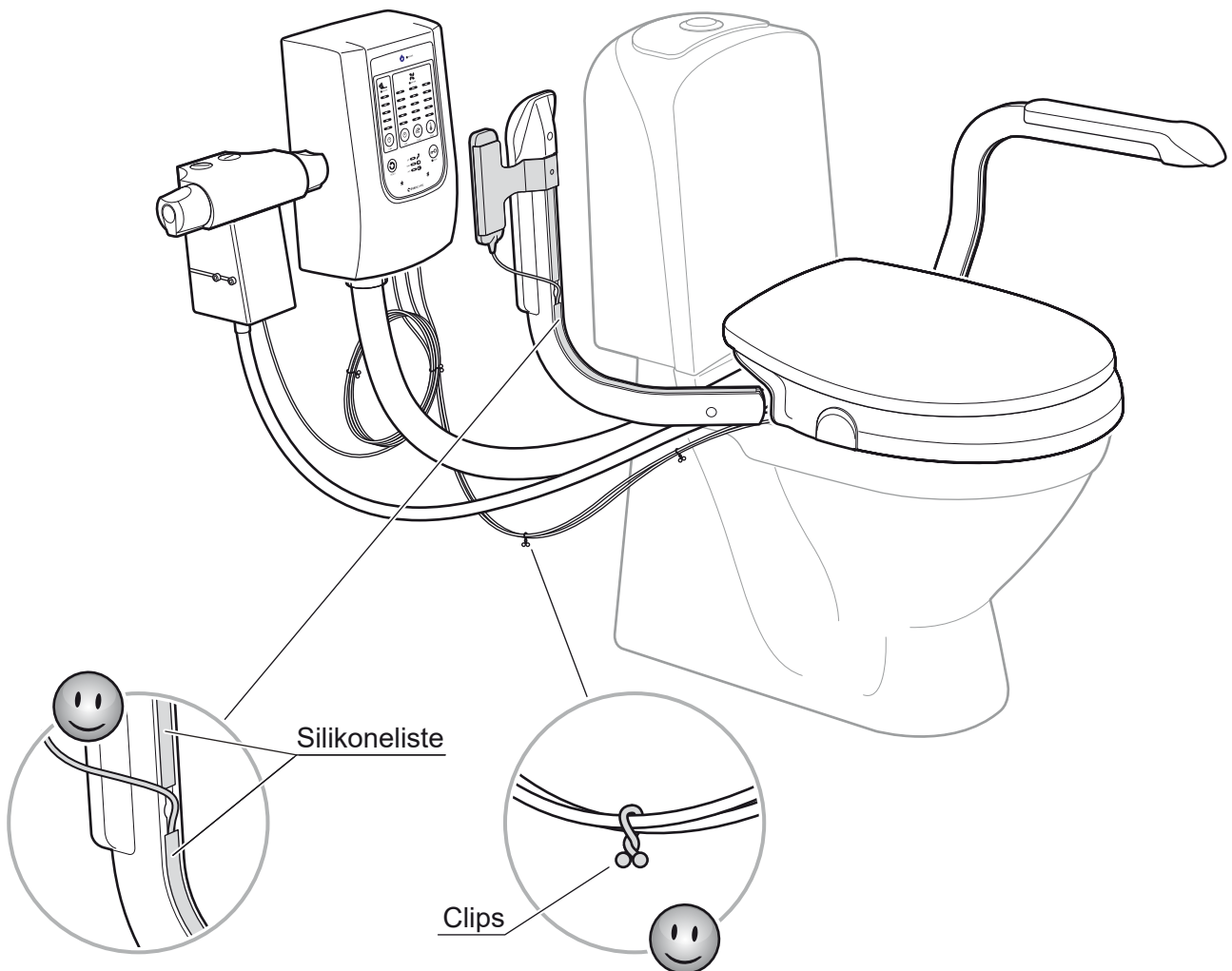
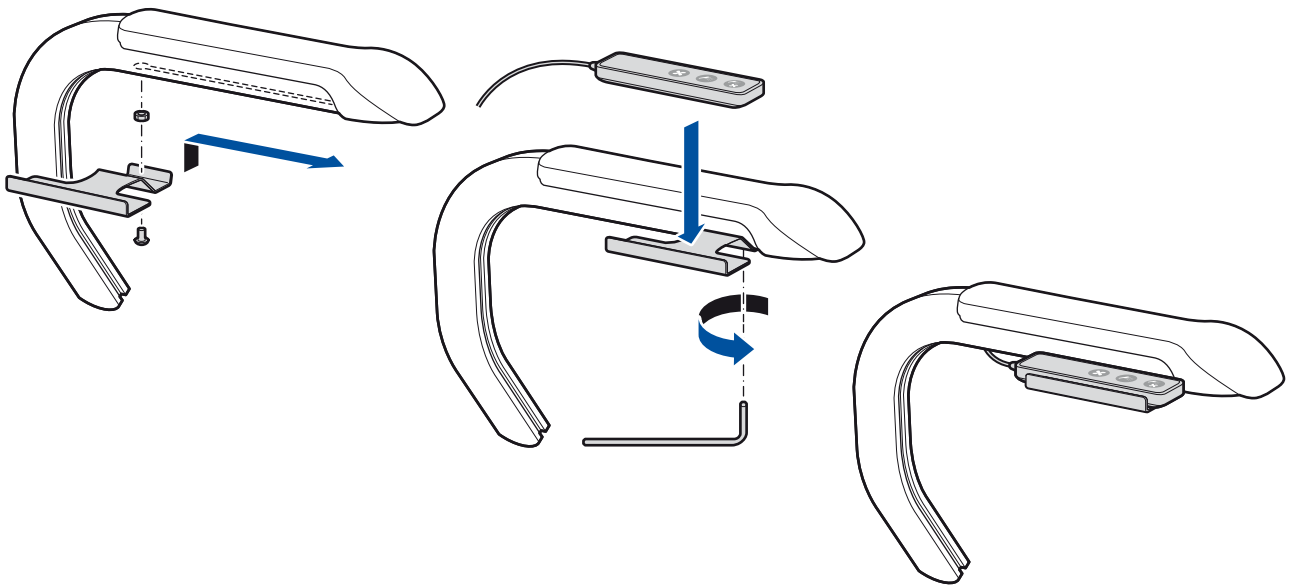
Armrest Bidette for Bidette R3, M3, P12

Item no. 11 401

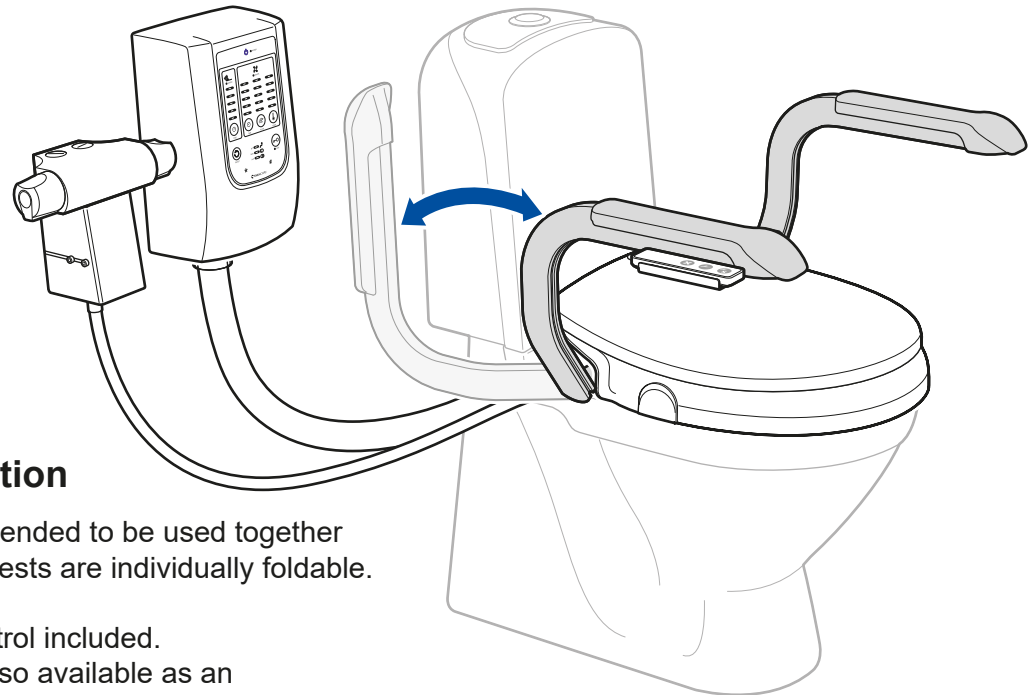Read **Installation and User Manual** prior to the installation and use of **Armrest Bidette**. Retain the manuals for future reference.

Product Description

Armrest Bidette is intended to be used together with Bidette. The armrests are individually foldable.

Holder for remote control included.
A toilet roll holder is also available as an accessory, to be installed under the grip.
Item no. 11 495

Armrest Bidette (installed on Bidette R3)

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sidorna 1 - 4

 **Montering og brugsanvisning**

siderne 9 - 12

 **Monterings- og bruksanvisning**

side 5 - 8

 **Installation and User Manual**

pages 13 - 16

Important!

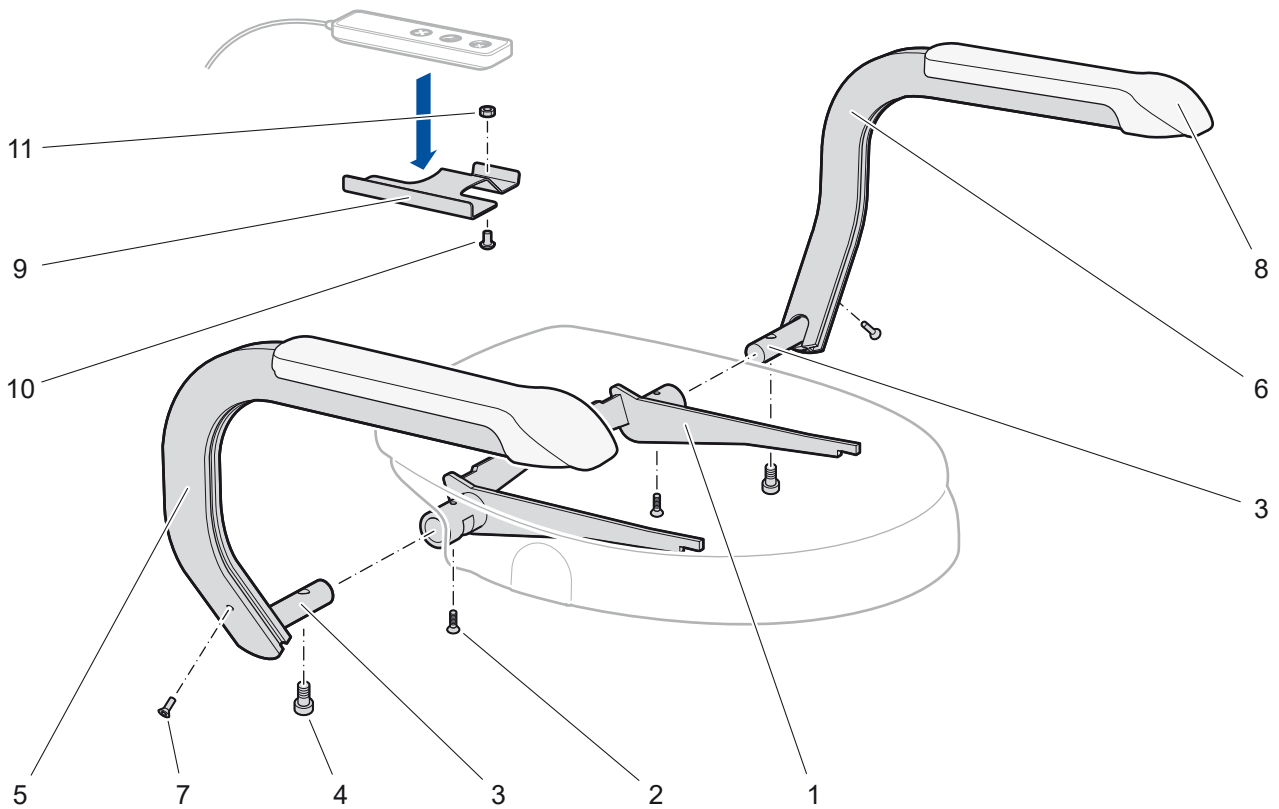
Armrest Bidette may only be installed on toilets that are well anchored to the floor or wall.

Use only original Svan Care accessories. Svan Care assumes no responsibility if the product is used, changed or installed in any manner other than that given in these instructions.

Max load: 50 kg/armrest, 150 kg/pair.
Estimated technical lifetime: 5 years

Exploded View

Svan Care reserves the right to make changes to the component list.
Current information can be found on our homepage.

Component list

No	Ant.	Designation	Item number
1	1	Frame	
2	2	Screw	11 183
3	2	Bracket	
4	2	Stop bolt	11 428
5	1	Armrest right	
6	1	Armrest left	
7	2	Screw	11 454
8	1	Grip armrest R3/ M3, piece	11 462
9	1	Holder for remote control R3 (pos. no. 10, 11 included)	11 480
10	1	Screw	included in 11 480
11	1	Nut	included in 11 480

Cleaning

As needed, clean the product with a mild detergent and water. Disinfect as needed.

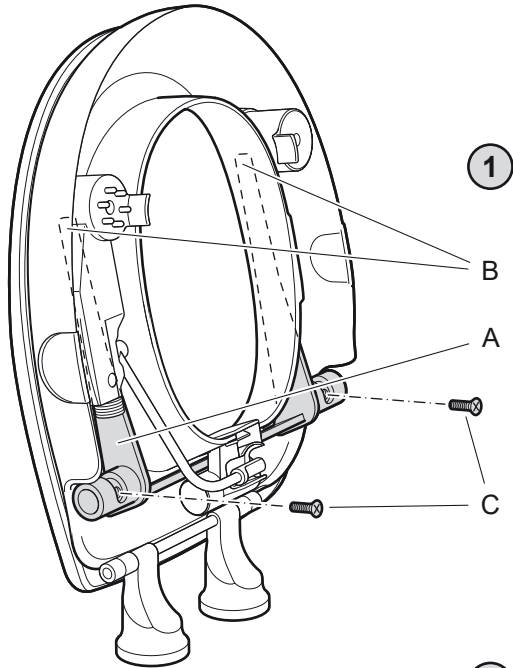
Reconditioning

Disassemble the entire product. Parts are washed clean with non-abrasive detergent, rinsed with water and dried. Disinfect. Check all parts and replace if worn.

Materials, environment and refuse sorting



The frame is made of lacquered stainless steel. The armrests are made of aluminium with grips made of ABS plastic. The grips are sent off for incineration or recycling. Metal parts are recycled as metal.

Installing the Armrest Bidette

1. Slide the frame (A) into the seat's two recesses (B) and fit the frame into the seat's two bowl-shaped brackets. Mount the two screws (C) in the respective bracket. **N.B.! Tighten the screws carefully, not too tightly.**

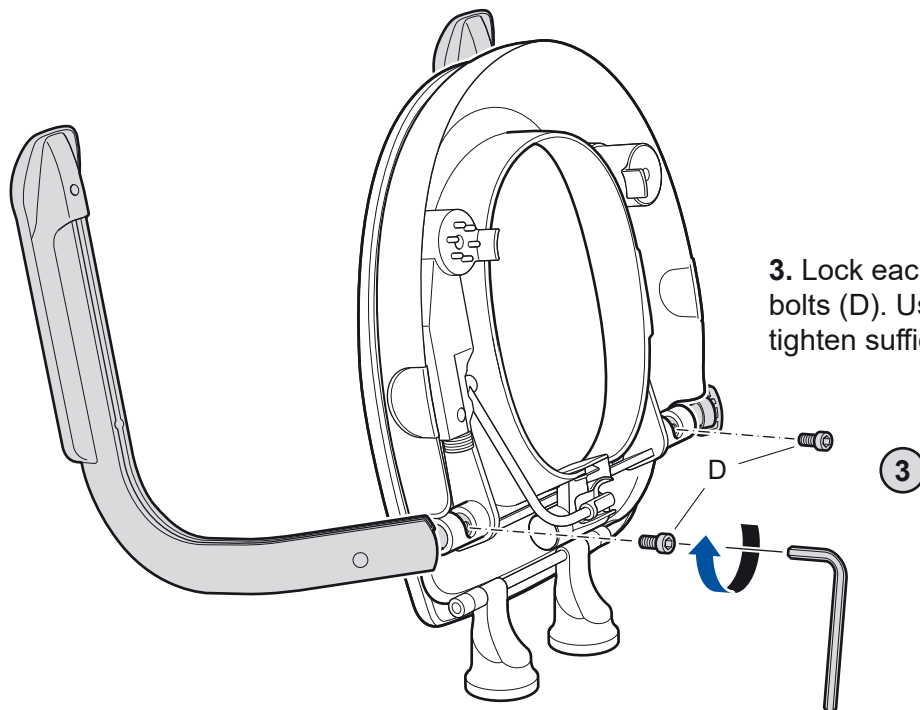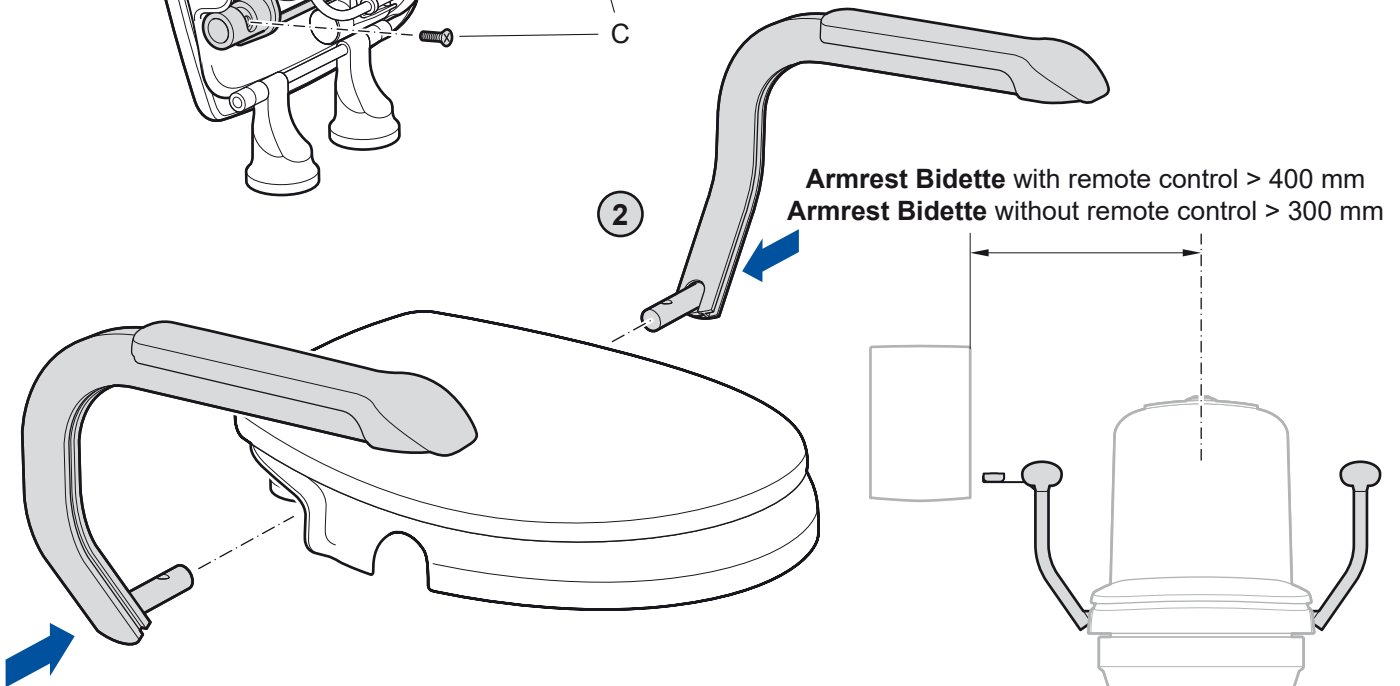2. Install right and left armrests on either side of the seat.

3. Lock each armrest with the two the stop bolts (D). Use the Allen key provided and tighten sufficiently.

Installation of holder for remote control on Armrest Bidette