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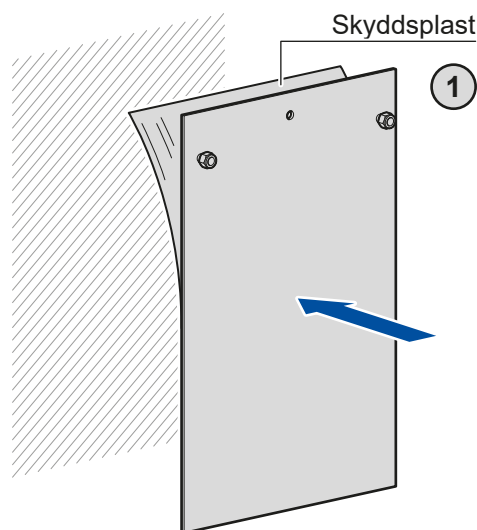
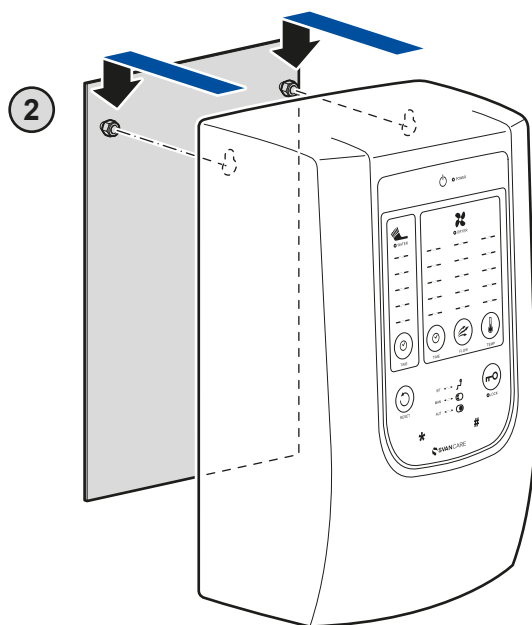
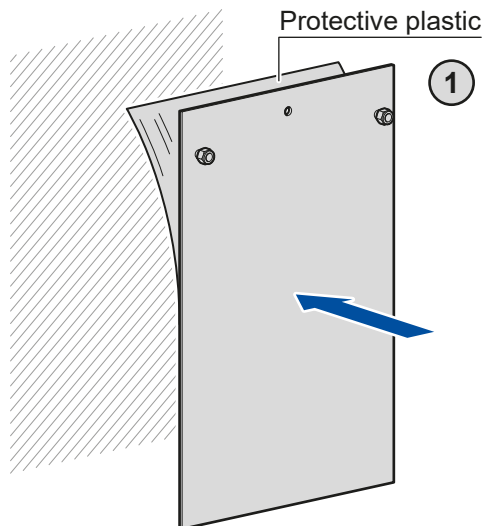
Mounting Plate Fan housing


Item no. 11 300

Read **Installation Instructions** prior to the installation of **Mounting Plate**. Save manual for future use.

Mounting Plate - Fan housing

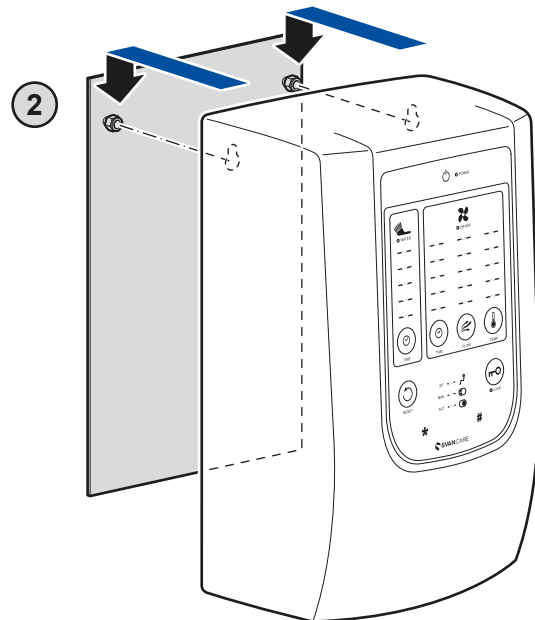Use

The **Mounting Plate** is used as a mounting accessory for the fan housing of Bidette when no unnecessary drill holes is acceptable.

Mounting

The **Mounting Plate** is provided with double-sided tape. The surface layer of the wall must withstand the load of the mounting plate and fan housing. Check that the wall can withstand this before installation. The surface must be completely smooth and clean. Tiles with a wavy surface (or similar) should be avoided. These reduce the contact surface with the tape, and the mounting plate will not adhere as firmly.

Mounting: Select a suitable smooth surface where the Mounting Plate will be installed. Clean the surface thoroughly with denatured alcohol and then with the included wipe.

1. Remove the protective plastic and press the plate firmly against the wall.

Manufacturer:

SVAN CARE AB
Alvägen 1
SE-771 41 LUDVIKA

Let the plate sit for an hour.

If there is a risk that the plate may come loose, the plate should be secured with a screw.

2. Mount the fan housing and check that it is hooked safely behind the two nuts.

Removal

When removing an attached plate, use a piano wire or similar to cut through the foam layer between the plate and the wall. Hold the piano wire with two pliers and saw downwards, placing a knife to follow along as the plate loosens. Remove the adhesive residue with a glass-ceramic scraper, and finally clean the surface with denatured alcohol.

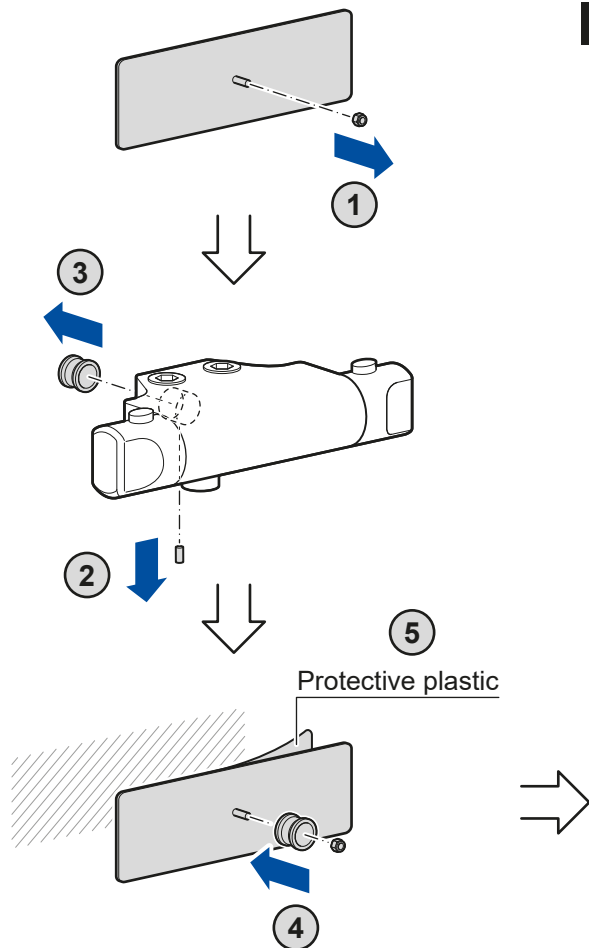
Cleaning

The **Mounting Plate** requires no special maintenance. It is made entirely of stainless steel. If necessary, clean the product with lukewarm water and mild detergent, or possibly with denatured alcohol.

Installation Instructions

Mounting Plate Thermostatic mixer

Item no. 11 300

Mounting Plate - Thermostatic mixer

Use

The **Mounting Plate** is used when installing a thermostatic mixer for Bidette and you don't want to damage the wall with unnecessary drill holes.

Mounting

The **Mounting Plate** is provided with double-sided tape. The surface layer of the wall must withstand the load of the mounting plate and mixer. Check that the wall can withstand this before installation. The surface must be completely smooth and clean. Tiles with a wavy surface (or similar) should be avoided. These reduce the contact surface with the tape, and the mounting plate will not adhere as firmly.

Mounting:

1. Remove the nut from the plate.
2. Remove the Allen screw from the mixer.
3. Remove the sleeve from the mixer.
4. Mount the sleeve and the nut on the plate.

Select a suitable smooth surface where the Mounting Plate will be installed.

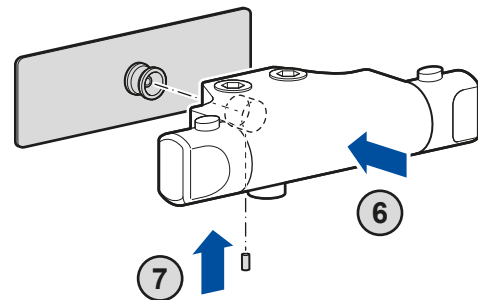
N.B.!

IMPORTANT!

A tiled wall with a wavy surface or similar does not work.

If there is a risk that the **Mounting Plate** may come loose, screws must be used. The wall where the **Mounting Plate** is to be mounted must be completely flat, smooth and free of grease.

Clean the surface thoroughly with denatured alcohol and then with the included wipe.

5. Remove the protective plastic and press the plate firmly against the wall.

Let the plate sit for an hour.

6. Mount the mixer on the sleeve.

7. Lock the mixer with the Allen screw.

Removal

When removing an attached plate, use a piano wire or similar to cut through the foam layer between the plate and the wall. Hold the piano wire with two pliers and saw downwards, placing a knife to follow along as the plate loosens. Remove the adhesive residue with a glass-ceramic scraper, and finally clean the surface with denatured alcohol.

Cleaning

The **Mounting Plate** requires no special maintenance. It is made entirely of stainless steel. If necessary, clean the product with lukewarm water and mild detergent, or possibly with denatured alcohol.