# HEPRO

---

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Manual

NORSK	2 - 3
SVENSKA	4 - 5
DANSK	6 - 7
SUOMI	8 - 9
ENGLISH	10 - 11
DEUTSCH	12 - 13
FRANÇAIS	14 - 15
ESPAÑOL	16 - 17
ITALIANO	18 - 19
NEDERLANDS	20 - 21

Montering av

## REKKVERKSØR

Art. Nr. 35149

## A

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
Art. No. 35149

## A


There are several ways in which the handrail can be installed, and this depends on area of use. In cases, e.g. of instalment of several poles in succession, in combination with the handrail tubes, a walking path can be created. The tubes can also be installed at different angles, in which case they can be used as handrails, e.g. in a staircase.

The contents in the delivery received therefore depend on the desired configuration.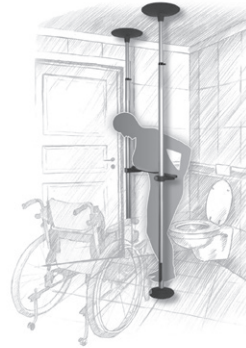

## B

When collapsible handrail tubes are installed between two Gripo poles, the accessory package will consist of two universal connectors and one handrail tube.

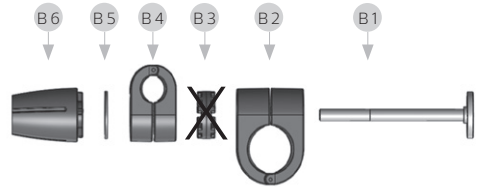
## C

The largest clamp **B2** is attached to the Gripo pole. For the smallest clamp **B4**, which also holds the handrail tube, to be rotatable, the tooth washer **B3** must be removed (do not use). Then the universal connector is reassembled.


D

Set the universal connectors at the desired height, then push the handrail tube in place in the clamp. Tighten the universal connector firmly in place.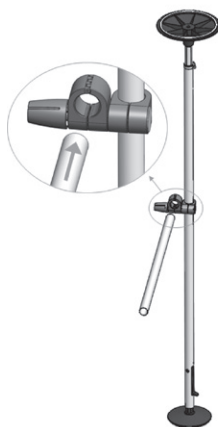

E

Place the universal connector with open grip at the desired height, then push the handrail tube into place in the clamp. The universal connector is tightened firmly.

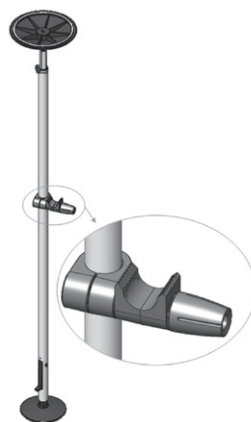Make sure the universal connectors are sufficiently tightened so that they do not slide down when the handrail tube is used.

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