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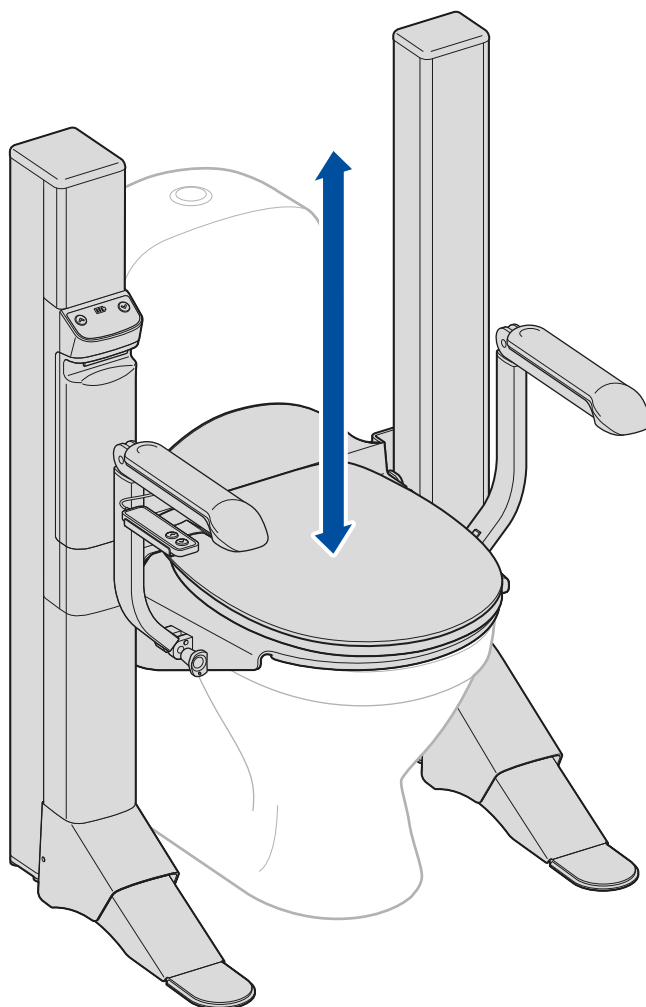
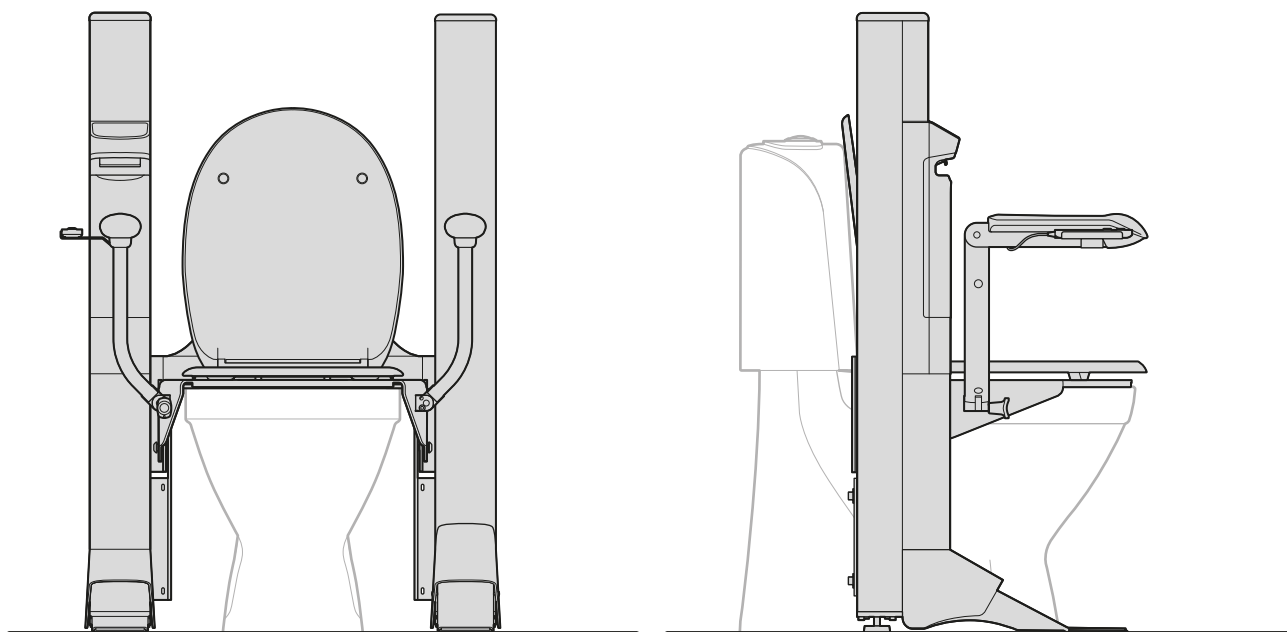
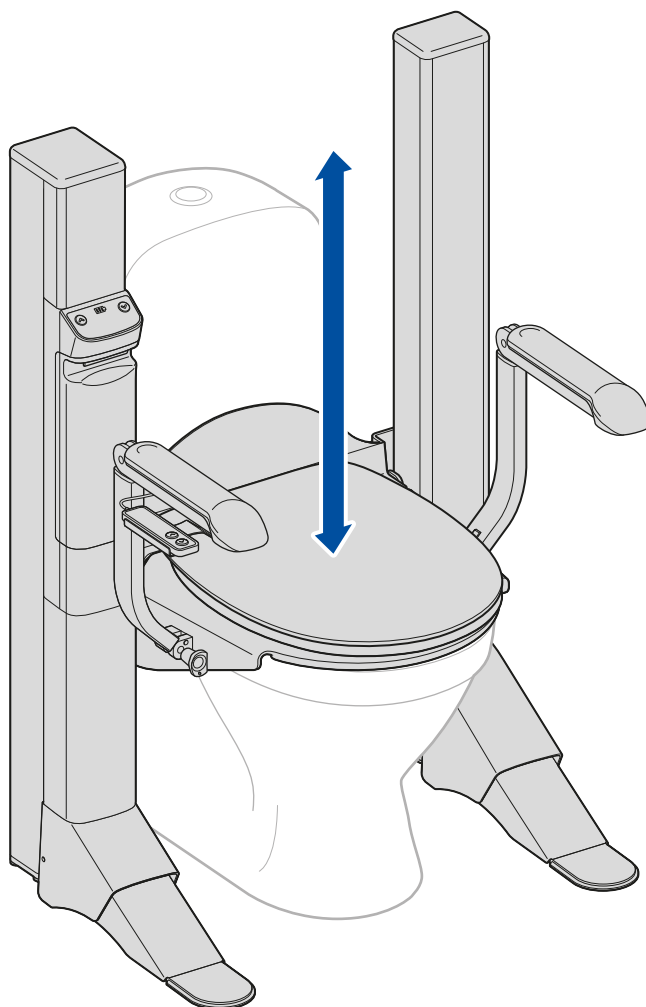
Read **Installation Instructions** and **User Manual** prior to the installation and use of **Lift Easy**. Save manual for future use.

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seiten 81 - 100

**Installation Instructions
and User Manual**

pages 101 - 120

Manufacturer:
SVAN CARE AB
Alvägen 1
SE-771 41 LUDVIKA

Phone: +46 24 01 01 02
info@svancare.se
www.svancare.se

Product description

LIFT Easy provides an aid for people who need help when moving to and from the toilet seat, and enables independence in connection to toilet visits.

LIFT Easy provides safe support from sitting to standing, as well as when moving between a wheelchair and a toilet seat.

LIFT Easy can easily be adjusted to lift in horizontal or tilted position to facilitate standing up.

LIFT Easy can be used as a shower area.

Clinical benefit

The clinical benefit of **Lift Easy** is to enable safe standing up from sitting on the toilet. It provides support when standing up as the seat follows the movement. The armrests also follow the movement, which makes it a stable and safe standing up. **Lift Easy** also makes it easier for those who have difficulty sitting at a low height. **Lift Easy** can also be used to spare the arms and shoulders when making low transfers. The clinical benefit is to provide independence in transfers to and from the toilet.

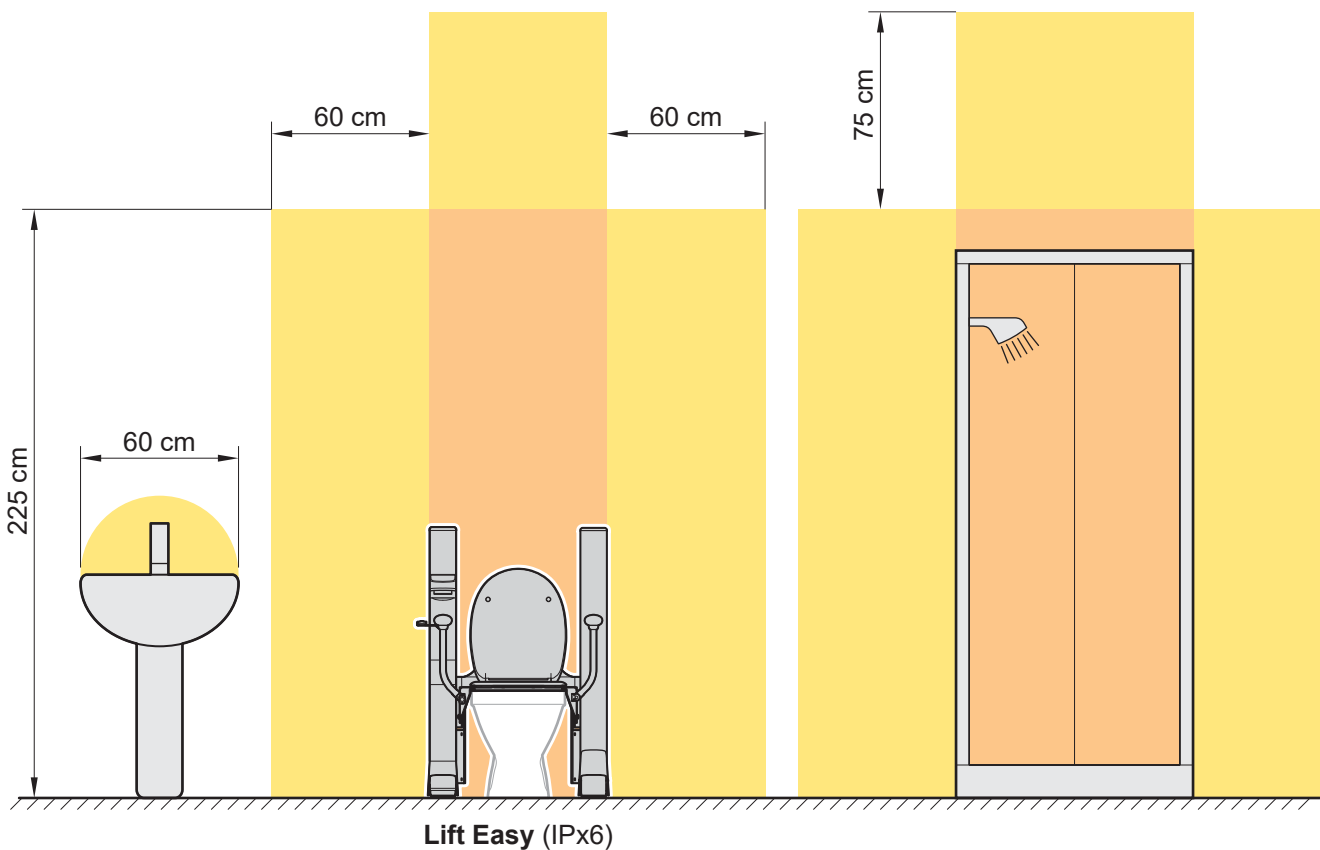
Lift Easy can be used without armrests if a qualified person assesses that the user is able to sit safely without armrests.

Lift Easy is developed and manufactured from sustainable materials by **Svancare** in Sweden and designed for modern bathrooms. The product is expected to last 8-10 years with normal use.

LIFT Easy is adapted for most toilet models and is designed for easy cleaning and can be mounted without any damage to the wall or floor in the bathroom.

If a serious incident occurs in connection with the use of **Lift Easy**, report the incident to the manufacturer and the relevant national responsible authority.

Lift Easy - IP Rating in Bathroom

Color	Area	Claim
	Zone	IPx0
	Zone 2	IPx1
	Zone 1	IPx4

LIFT Easy – Technical specification

Nominal voltage:	200-240 VAC, 50/60 Hz
Power:	Max 240 W
Current:	1 A
Fuse:	Min 10 A
IP class remote control, battery module, other:	IPx6
Isolation class:	Class II
Applied parts:	Typ BF
Max lifting capacity:	200 kg
Max load armrest:	75 kg/armrest (150 kg/pair)
Max lift height (above floor):	950 mm
Operating mode	Temporary use
Technical lifetime:	5 years

Product labelling

The product is approved and CE-marked in accordance with MDR 2017/745.

Demonstrates compliance with Council Directive 2012/19/EU on waste consisting of or containing electrical and electronic equipment (WEEE). The symbol means that discarded electrical and electronic equipment must not be disposed of as household waste but must be turned in for separate recycling.

Applied Part Type BF.

Read Installation Instructions and User Manual.

The manufacturer's item number on the product.

Product serial number.

Indicates the date of manufacture.

Indicates the name and address of the manufacturer.

LIFT Easy meets insulation class II (double insulated).

Caution: This symbol indicates that caution should be taken in order to prevent injury to patients or persons and/or damage to the equipment.

Warning: The warning symbol indicates a potential danger which, if not avoided, may result in a serious injury to patients or persons.

Warning!

Warning!

The product must not be opened or repaired by unauthorized persons. Contact the manufacturer or your distributor for service assistance.

There is a risk of crushing injuries (personal injury) between the **LIFT Easy** and the toilet.

Make sure that the area around the **LIFT Easy** is free of furniture and other fixed objects.

NOTE! Applies to both raised and lowered positions.

LIFT Easy must not be used while it is being repaired or maintained.

Do not sit on the armrests (max. load: 75 kg per armrest).

Your feet should always be in contact with the floor during use.

Caution!

Caution!

Svancare is only responsible for safety, reliable operation and performance if the product is used according to the instructions given and the user follows the safety information in these instructions for use. Anyone who neglects this safety information exposes themselves, the patient and other personnel to potential danger.

Read the instructions for use carefully and follow them.

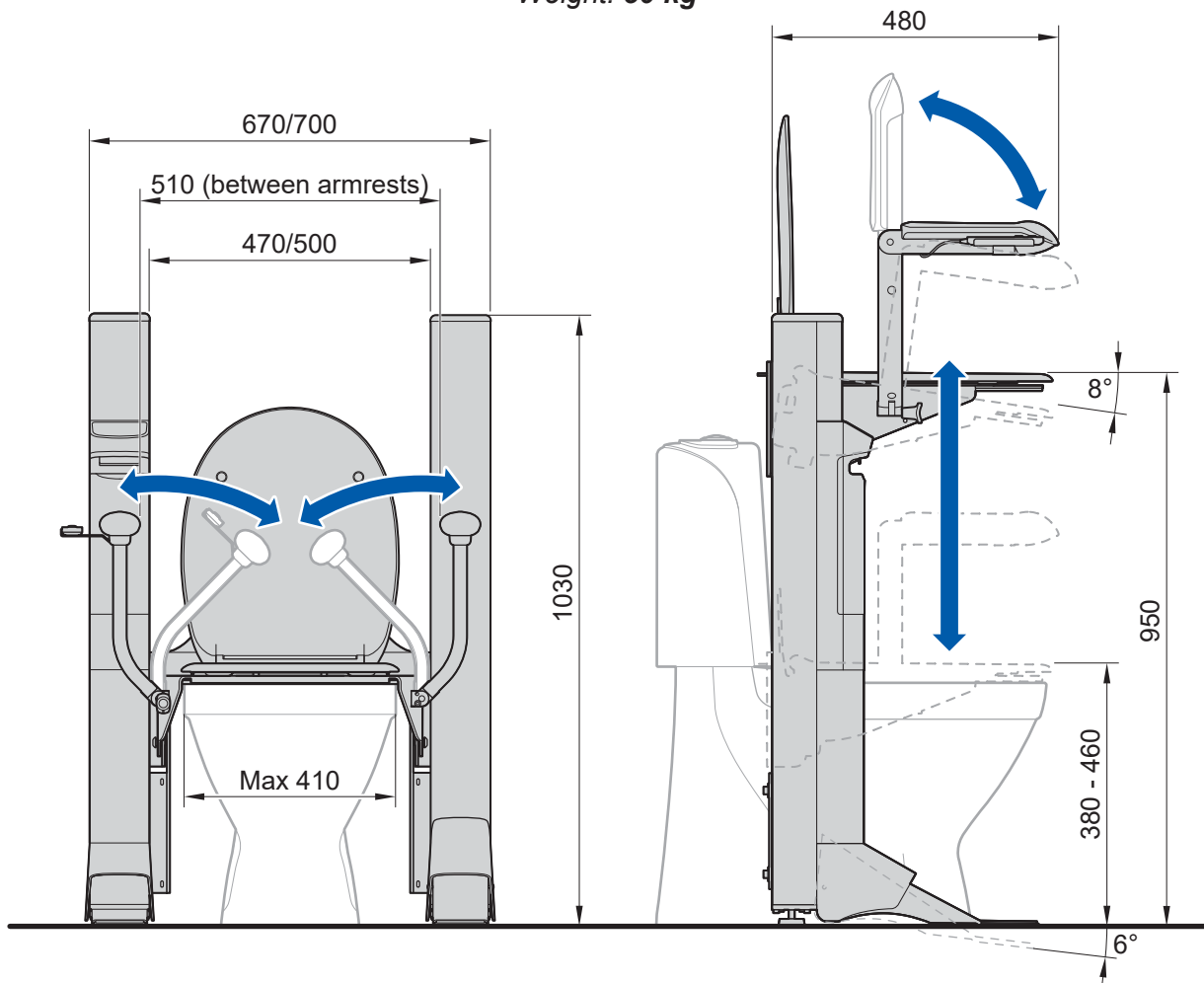
Children must be supervised to ensure that they do not play with this product.

The product must not be opened or otherwise modified by anyone other than **Svancare** or by **Svancare** trained personnel.

Make sure that the person who will use **LIFT Easy** understands how **LIFT Easy** works and that the person is able to sit stably on **LIFT Easy**.

Svancare assumes no liability if the product is used, altered or assembled in a manner other than that specified in these instructions for use.

Measurements (mm) LIFT Easy
Weight: 50 kg

Important to check before starting installation of LIFT Easy

Important! Before starting installation, check that there is free space around and above where **LIFT Easy** is to be mounted. Otherwise, there is a risk of crushing injuries. Ensure that **LIFT Easy** is level and stable using the adjustable feet on the pillars.

NOTE! The battery and control box, as well as the connection to the mains cable and remote control are located in **pillar A** (see dimensional drawing below). **Place pillar A on the side of the toilet that is most suitable for servicing the battery and installing the power cable.**

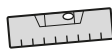
NOTE! You can choose between two different widths of **LIFT Easy** (see measurement drawing on page 105). Choose the width that best suits your installation.

NOTE! There is a risk of crushing in the area above **LIFT Easy**. Check that no shelves or similar are mounted in this area!


Tools for installation


Allen key No. 3, 4, 5, 6



Level

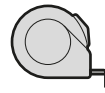Measuring tool

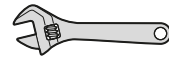
Wrench

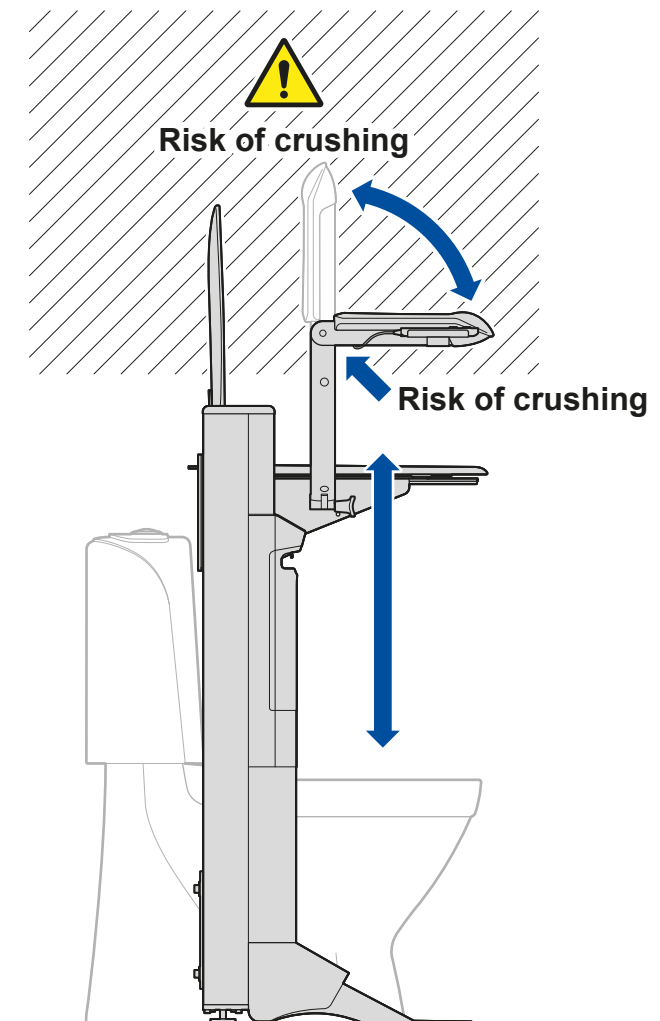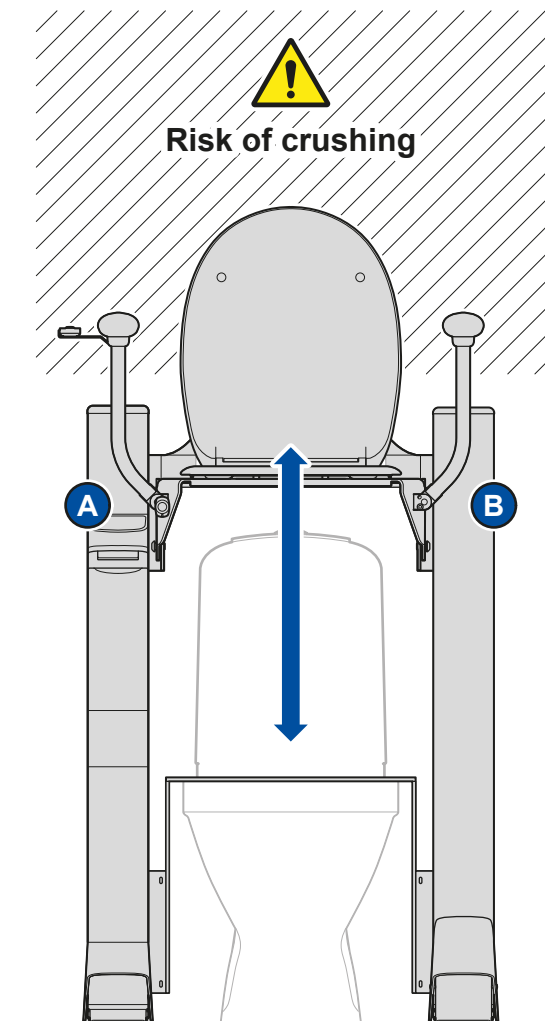
1

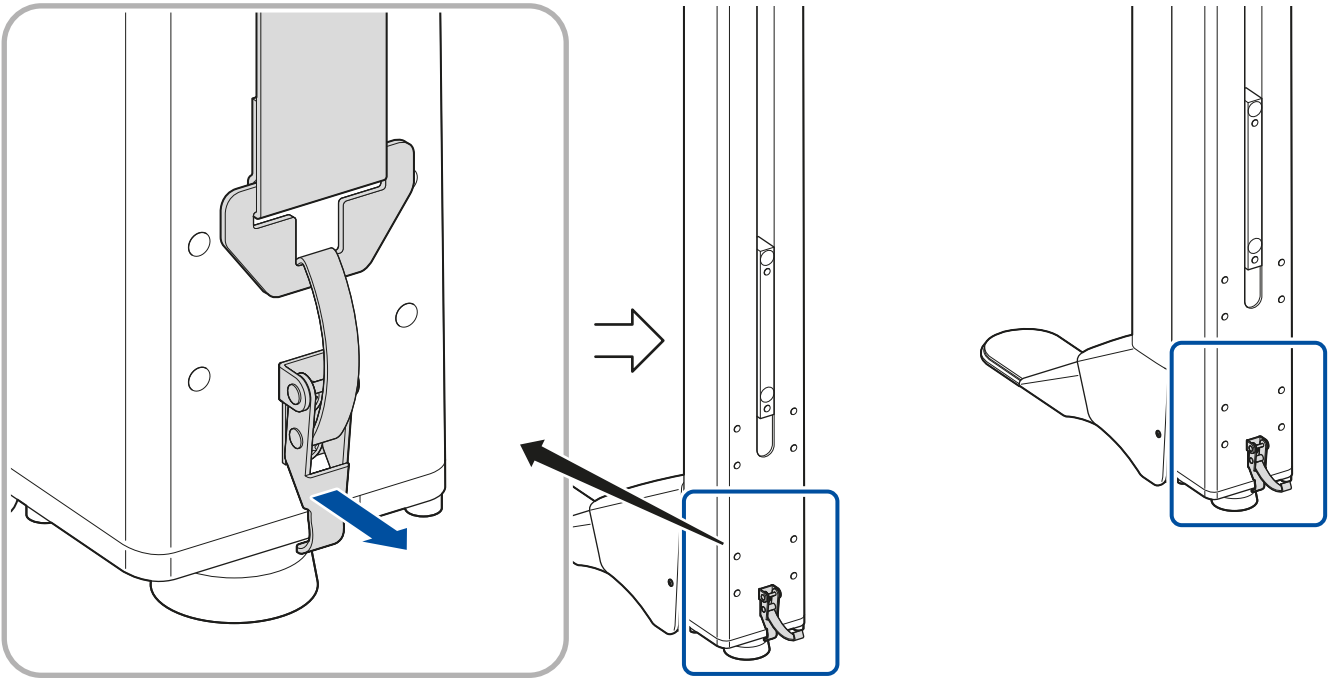Loosen the protective straps from both pillars


2

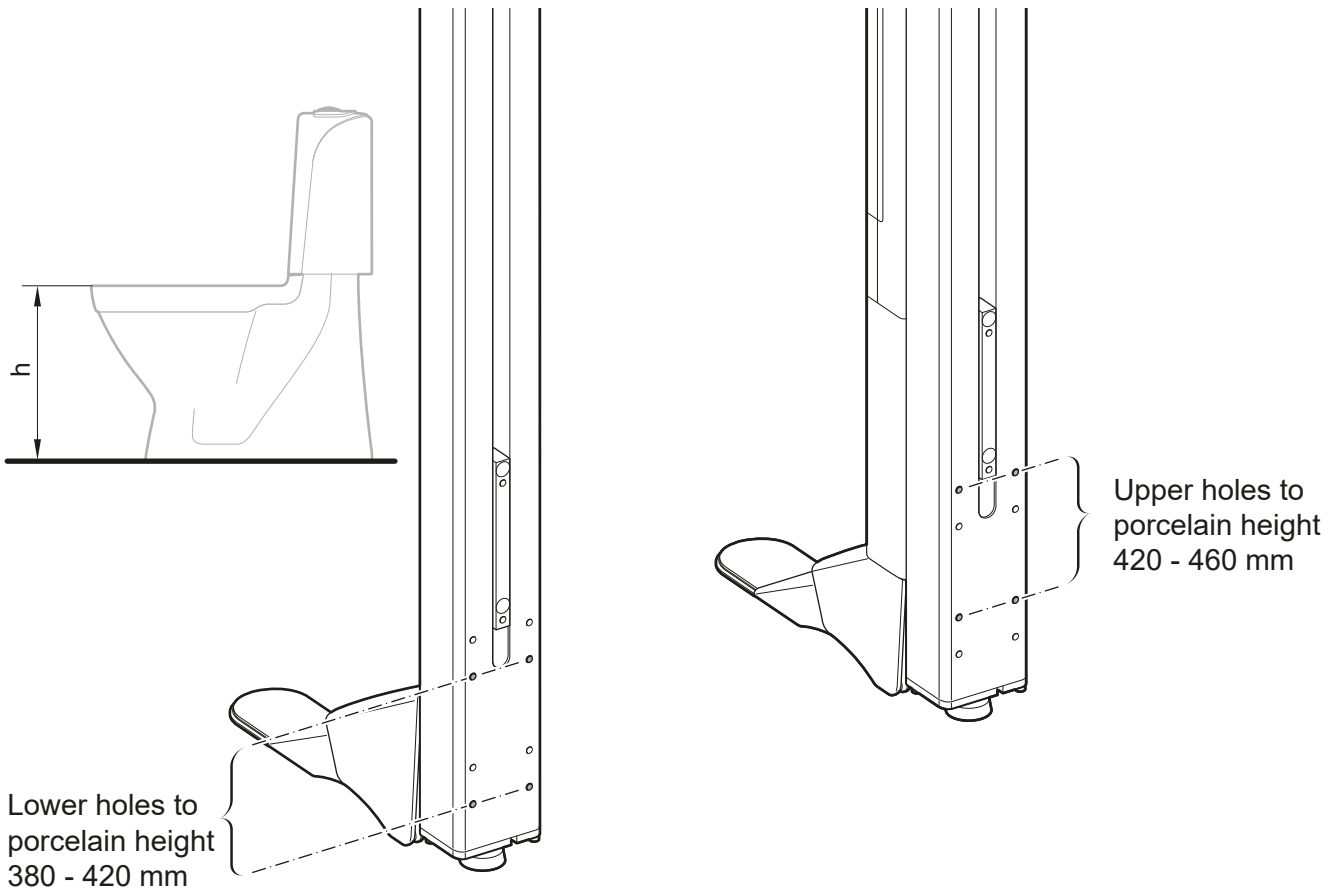Check the height "h" of the toilet porcelain


5

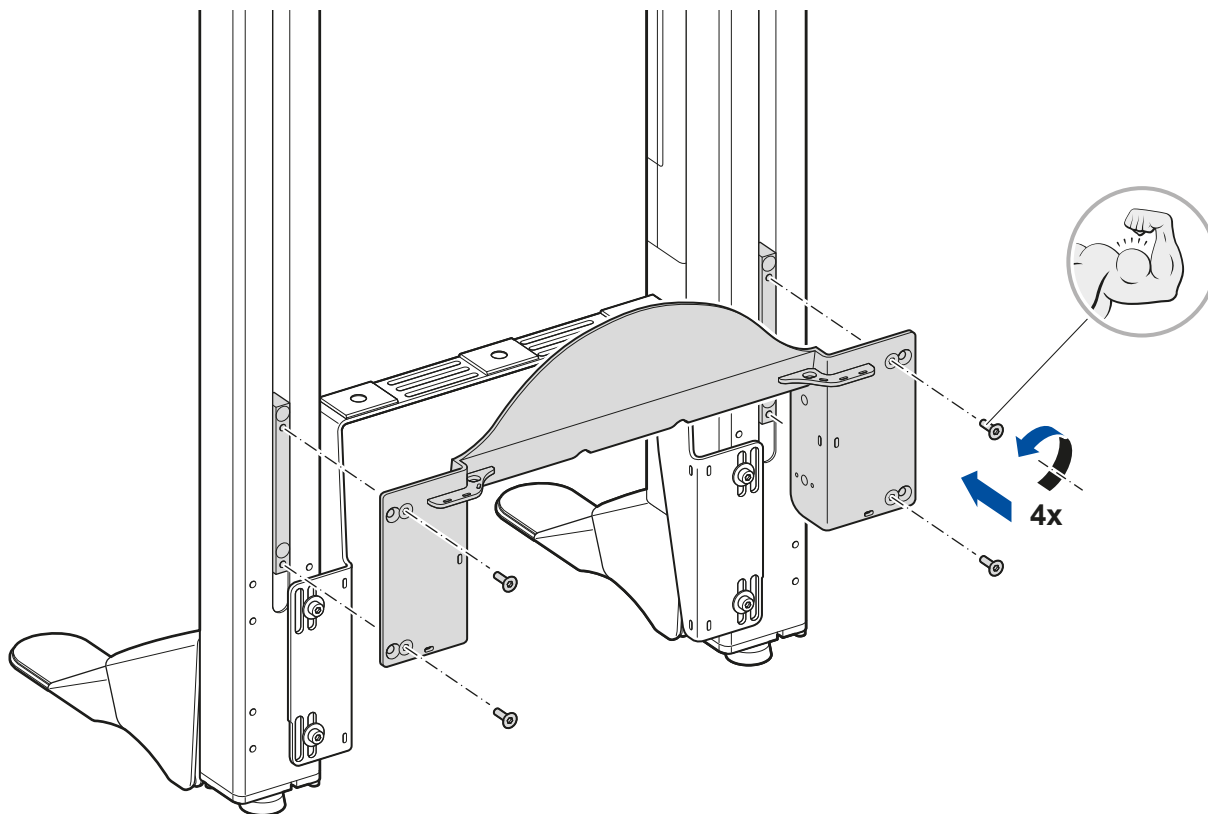Mounting of movable bridge in the pillars


6

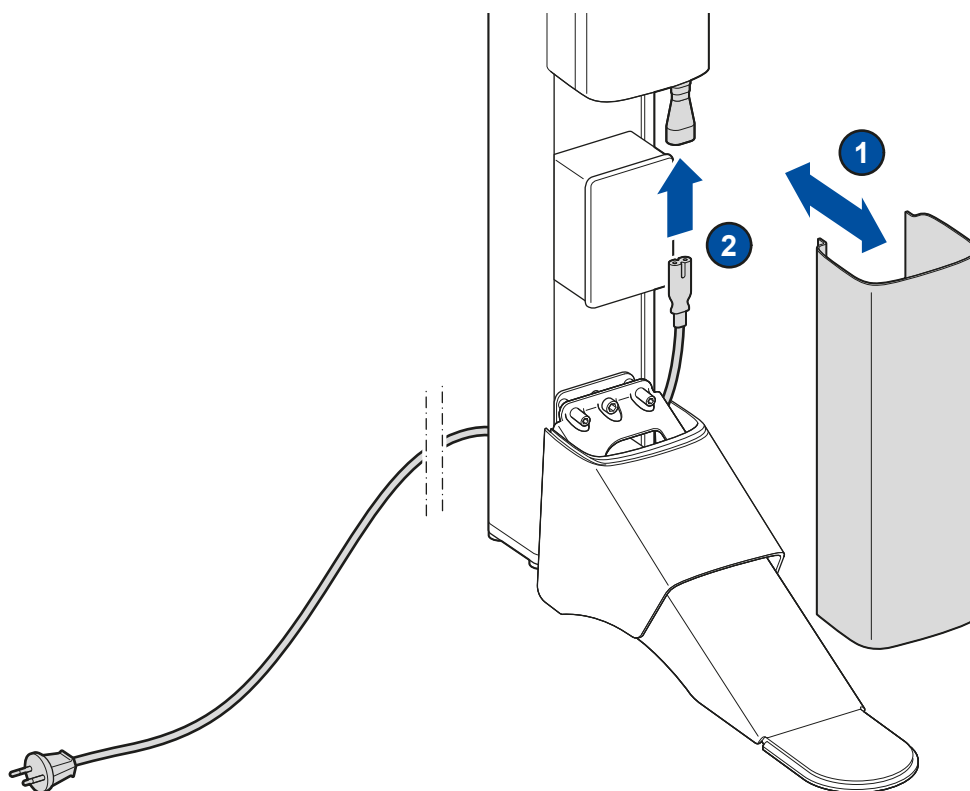Connect power cable to pillar A


7

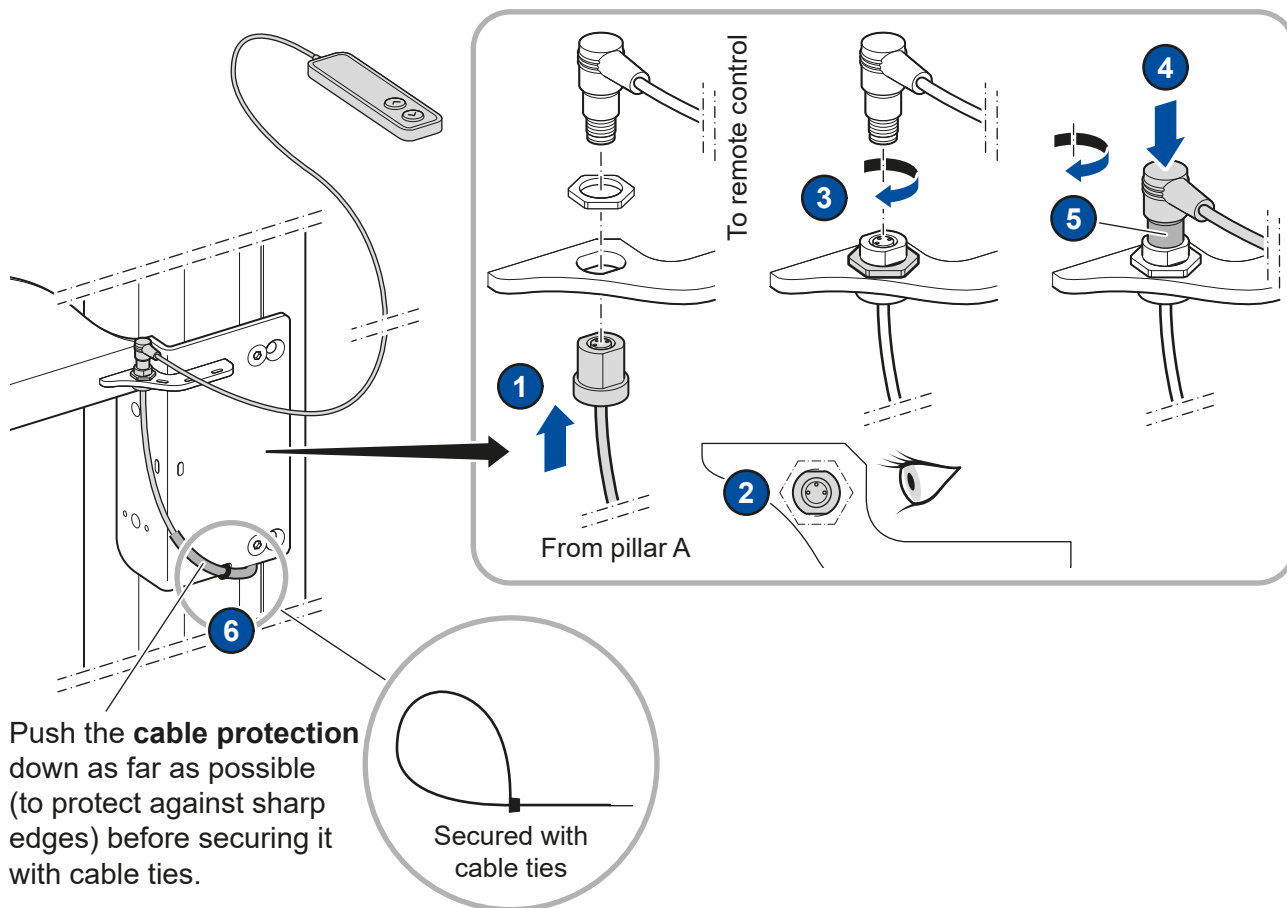
Connect remote control to pillar A

8

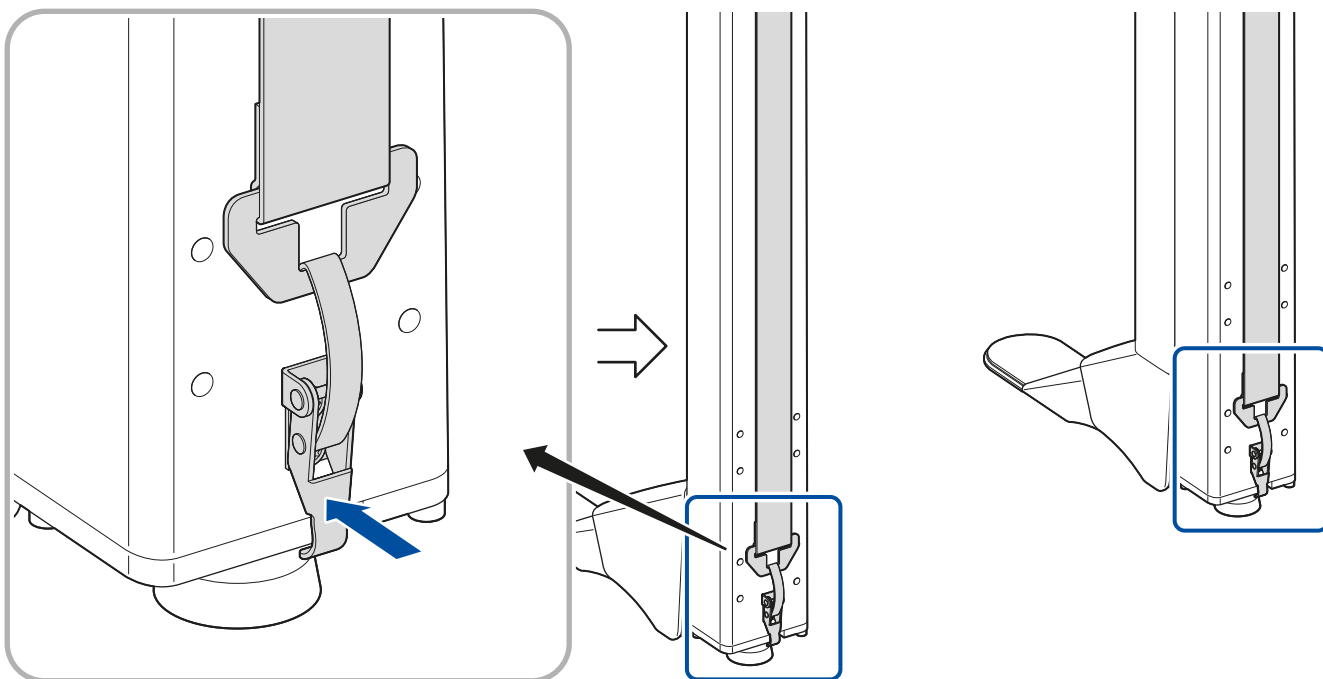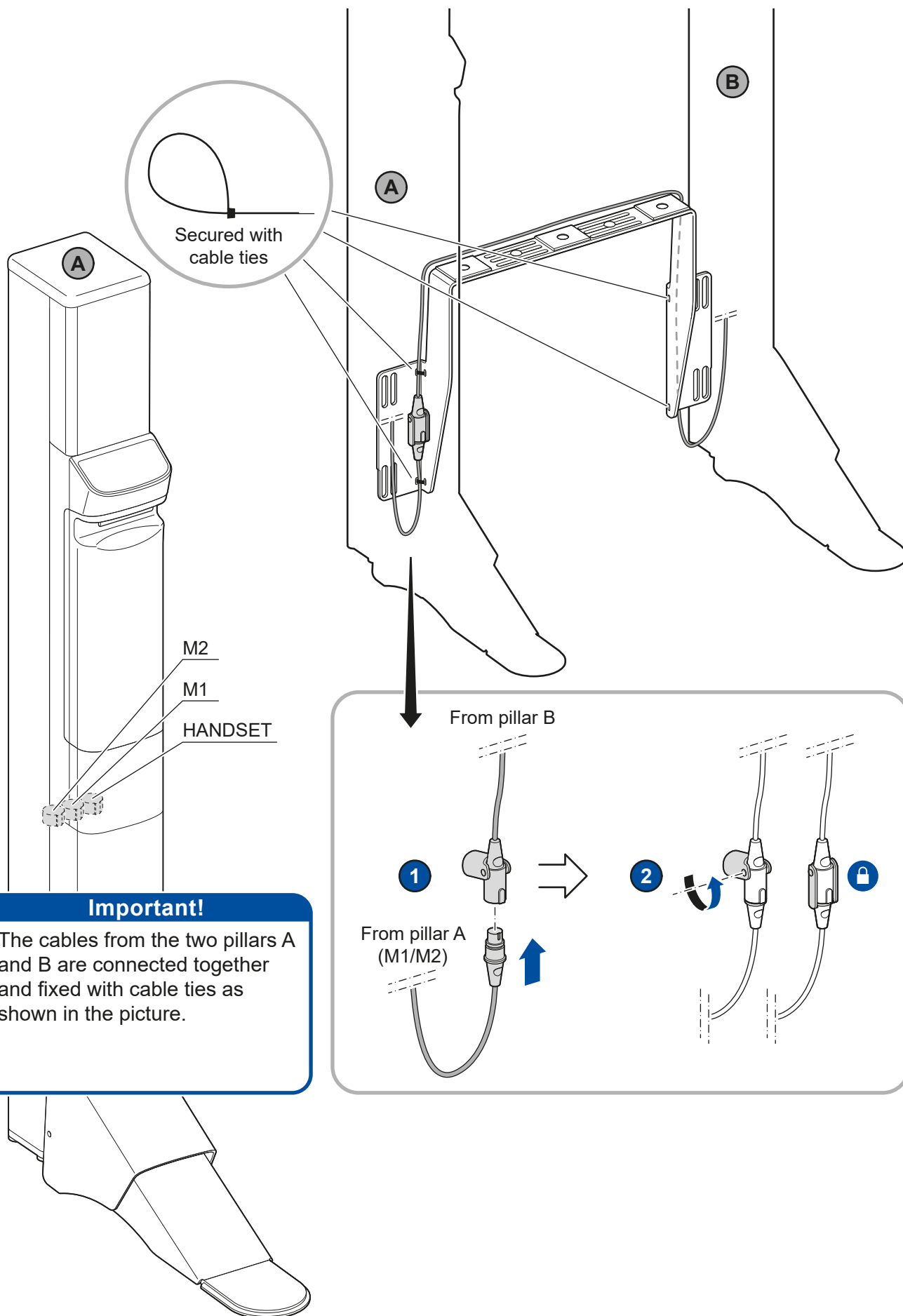Mount the protective straps to both pillars

**Important!**

The cables from the two pillars A and B are connected together and fixed with cable ties as shown in the picture.

N.B.!

Place **LIFT Easy** centered over the toilet.

1-2. Install screws, washers and nuts (bushes for stabilization if necessary).

3. Remove cover **A**.

4. Remove cover **B** (velcro tape).

5. Loosen the locking screw.

6. Adjust the feet on the two legs (3 set screws/leg) so that **LIFT Easy** stands level and stable (use a level).

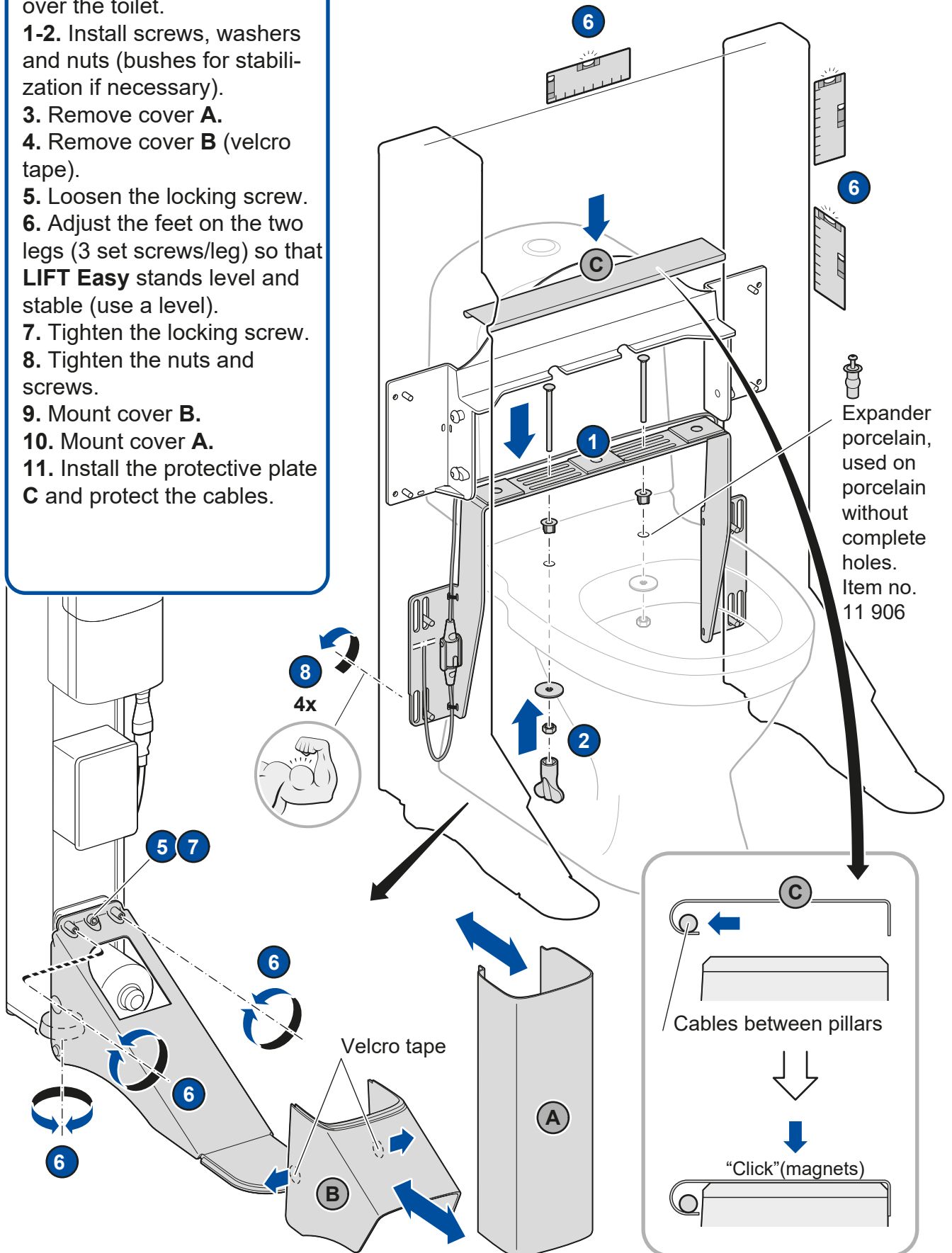
7. Tighten the locking screw.

8. Tighten the nuts and screws.

9. Mount cover **B**.


10. Mount cover **A**.

11. Install the protective plate **C** and protect the cables.


11

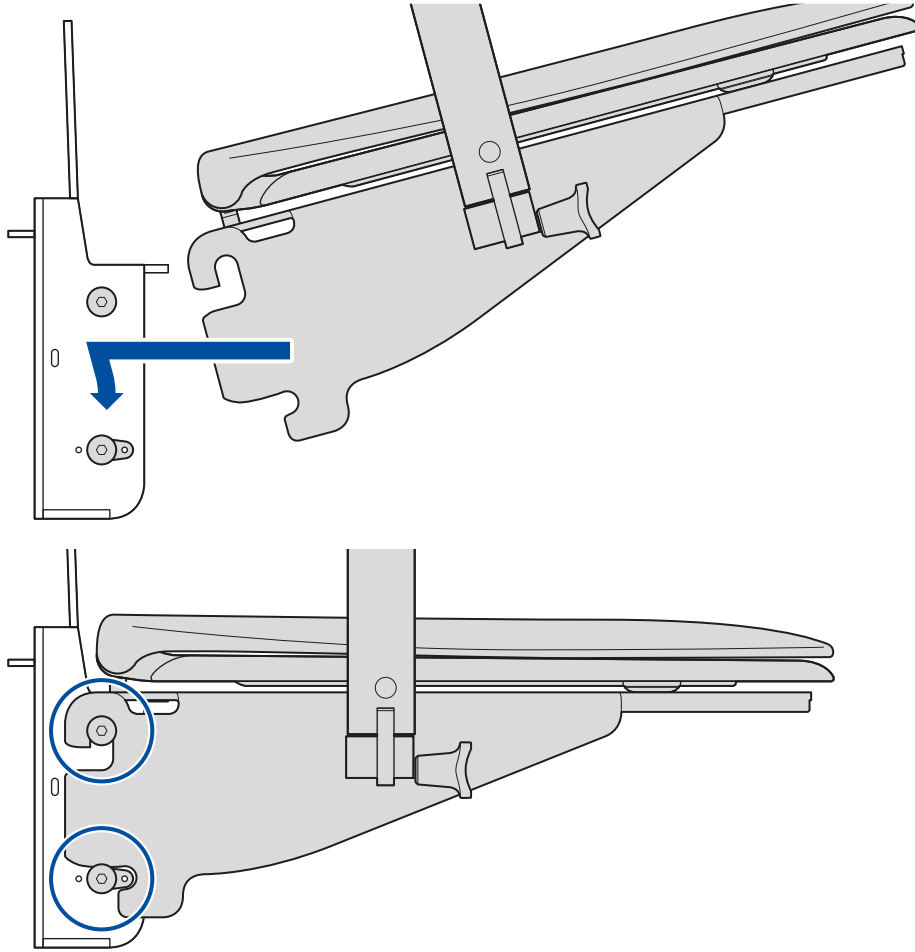Install the seat plate, seat and armrests


12

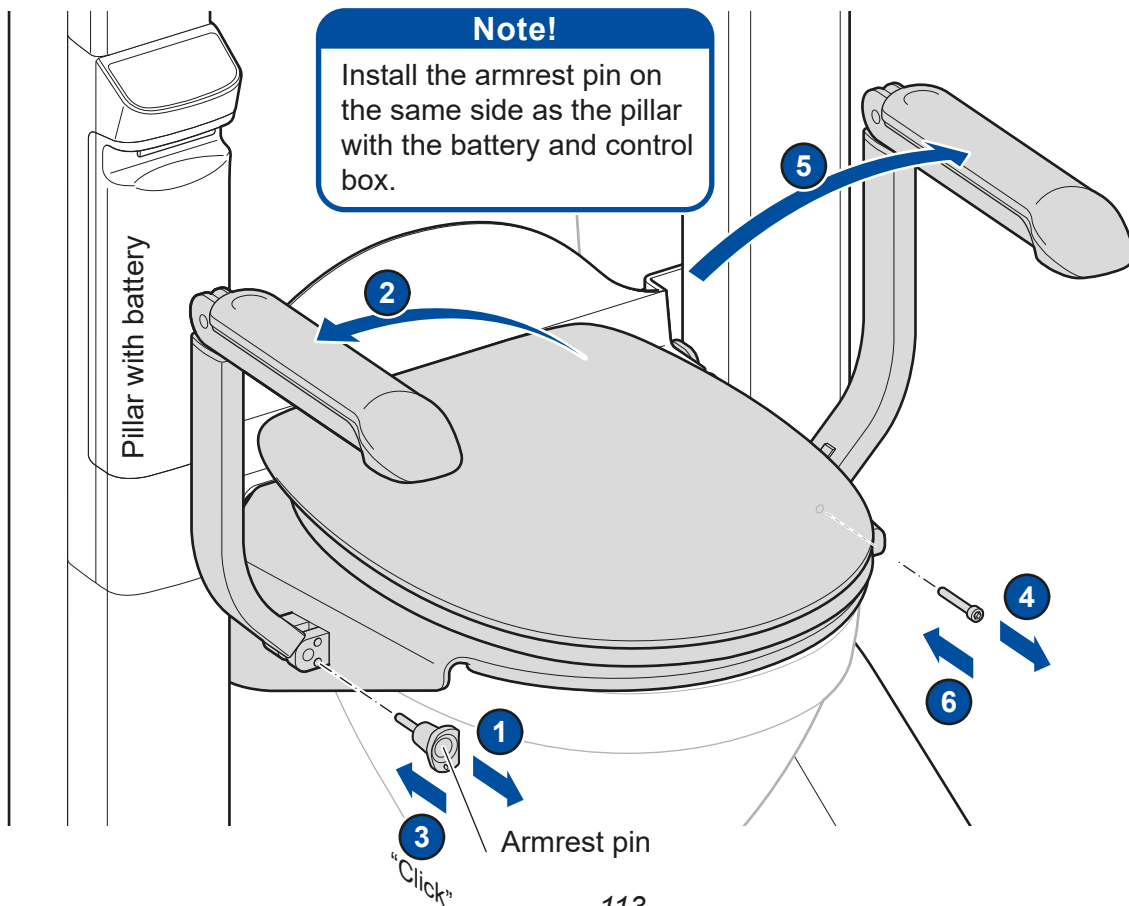Unfold the armrests


13

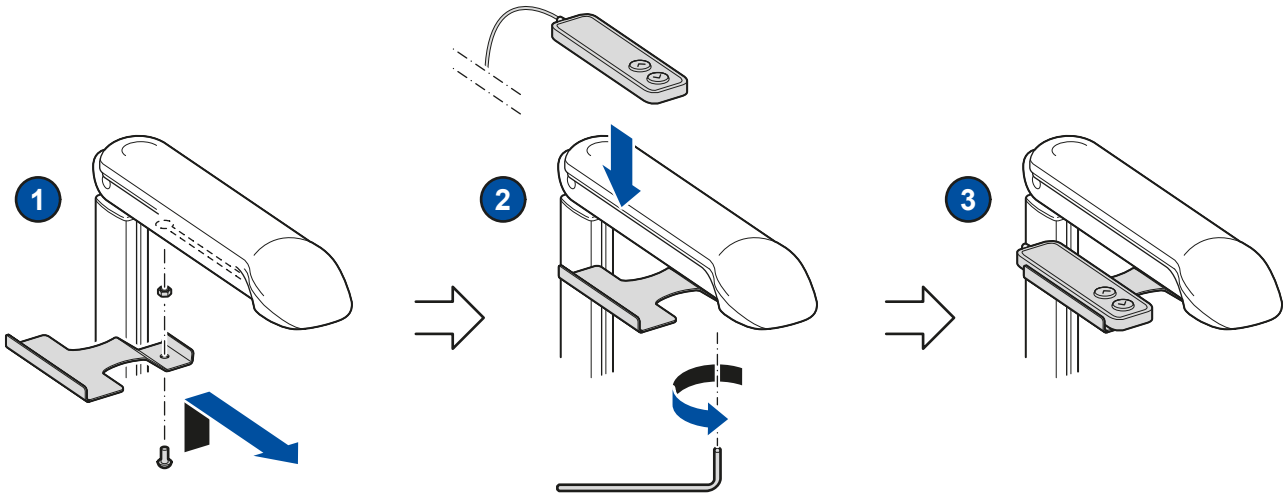
Mounting the remote control on the armrest (magnet)


14

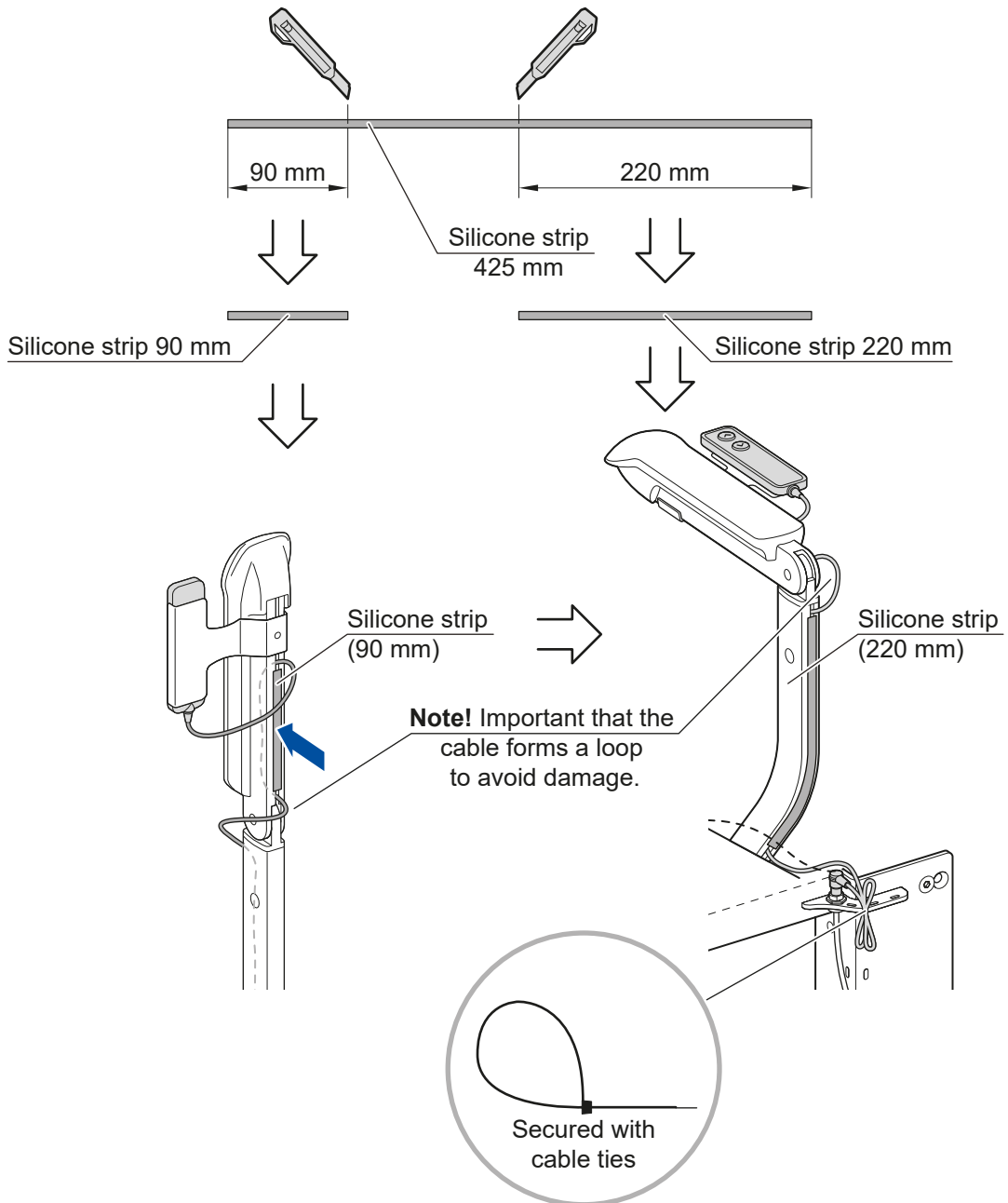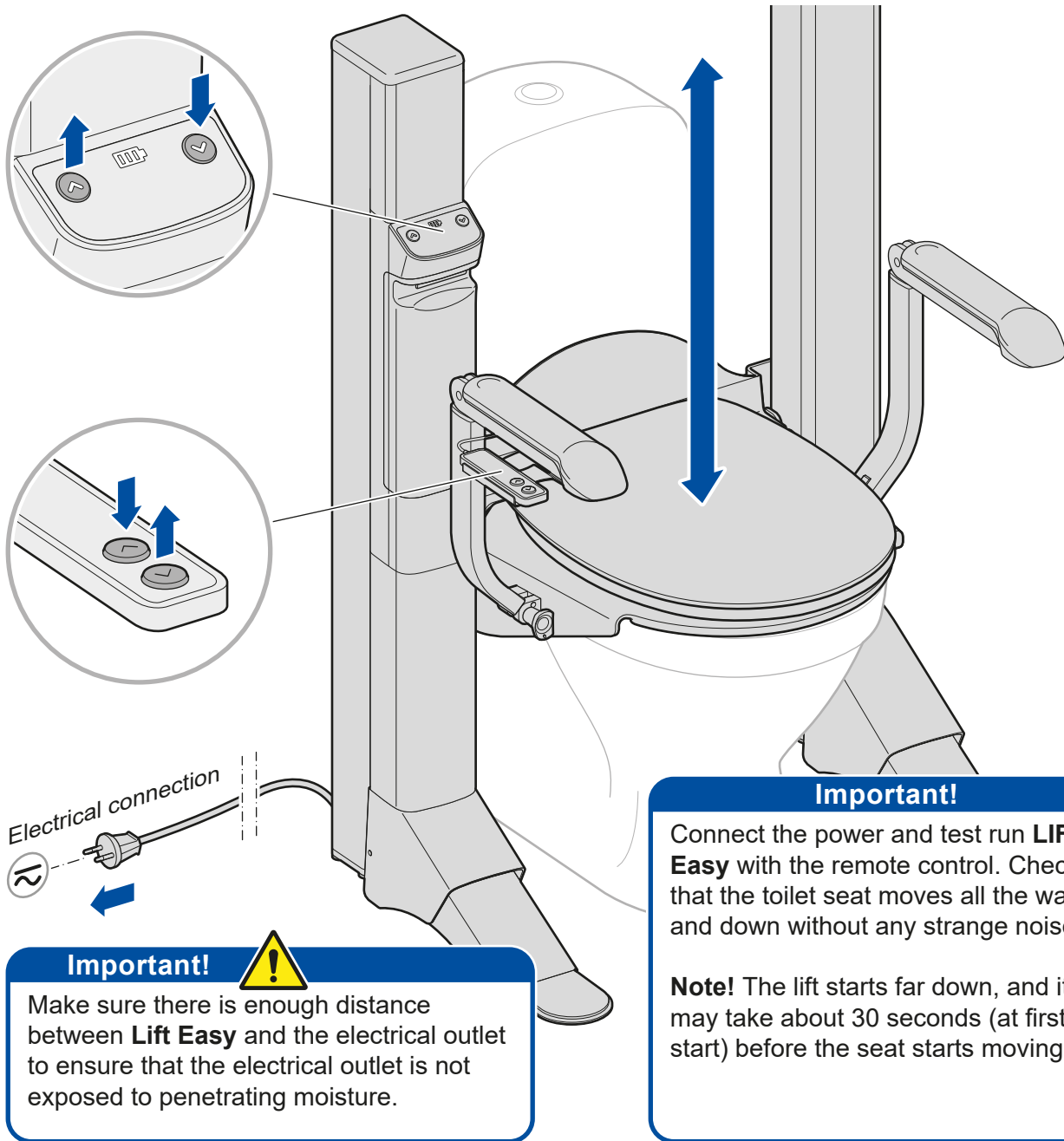Cutting silicone strip. Protection and routing of cable to remote control

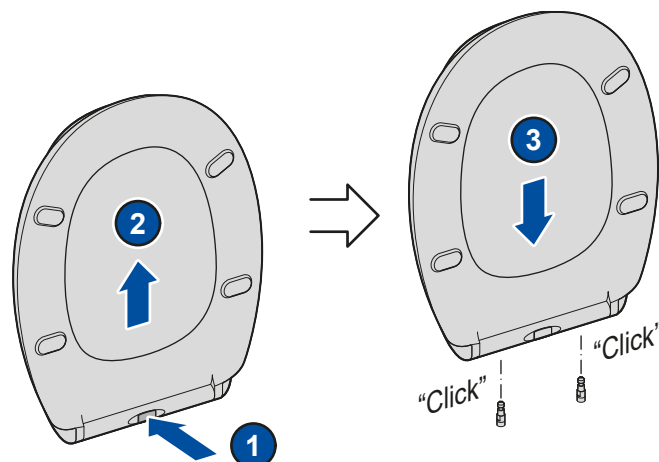
Toilet seat (soft-closing with quick release)


LIFT Easy - Safe support, up and down

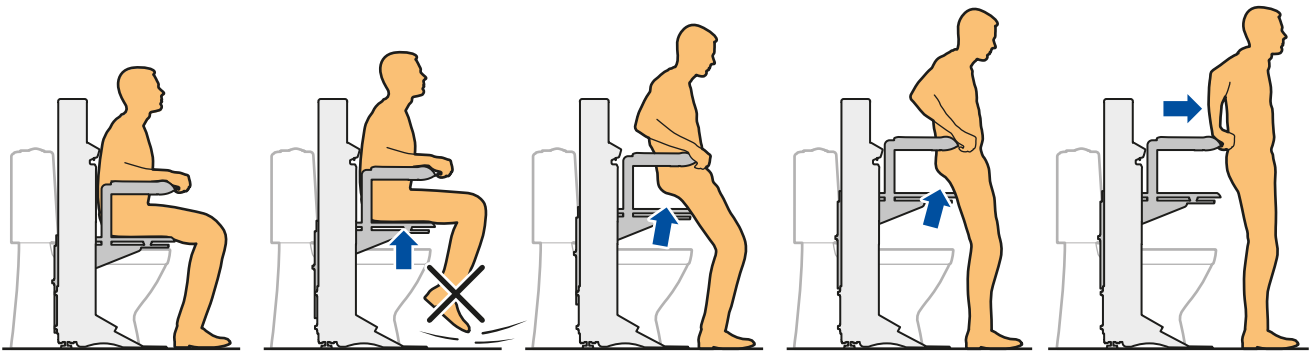LIFT Easy lifts people with weak knees and leg muscles to a standing position. Thanks to the contact between the toilet seat and the tail, the lift is safe and stable all the way to a standing position.


NOTE! Always remain in contact with the floor.

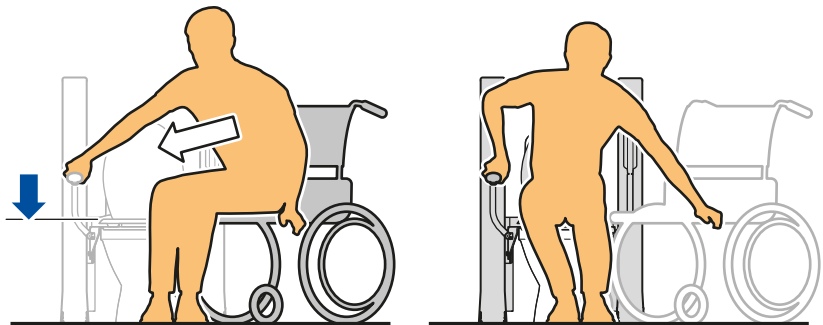
Side movement between wheelchair and LIFT Easy - always downwards

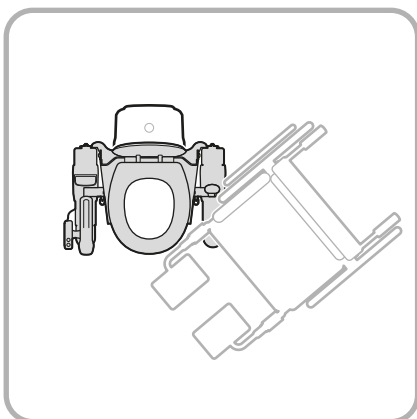From wheelchair to **LIFT Easy**
- side movement downwards

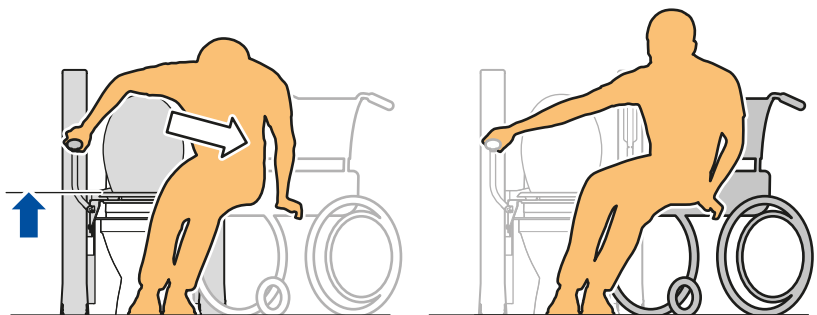Important to place the wheelchair close to **LIFT Easy** to facilitate side movement.




From **LIFT Easy** to wheelchair
- side movement downwards

Removing/installing battery


NOTE!

Pull the armrest pin out completely. Tilt the armrest and then remove the battery (1, 2, 3).

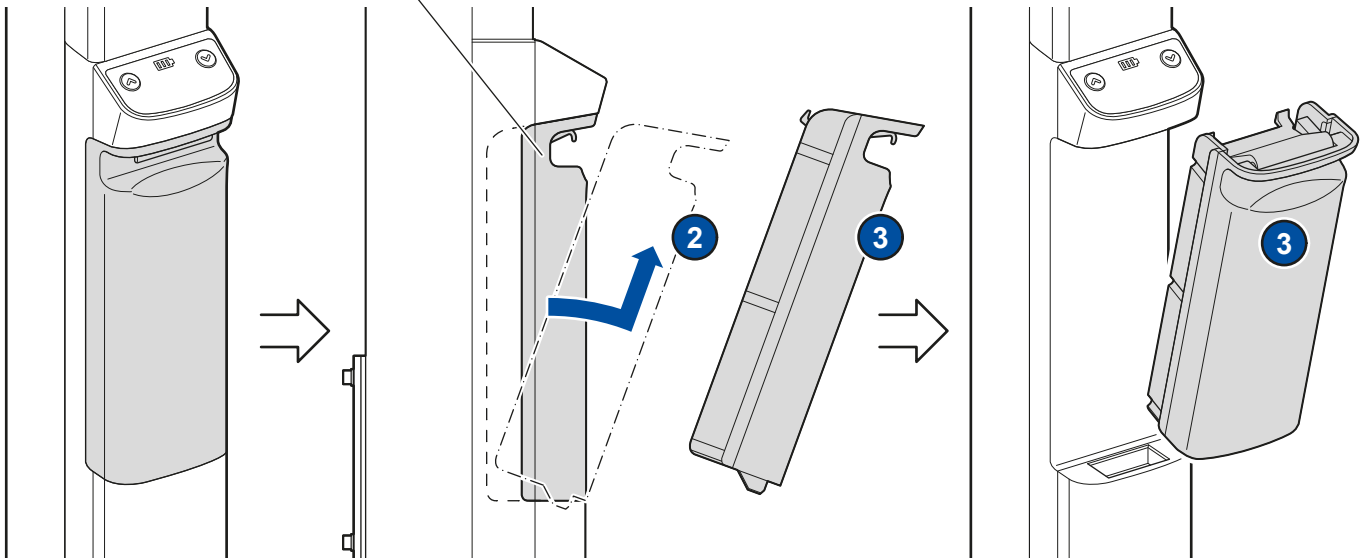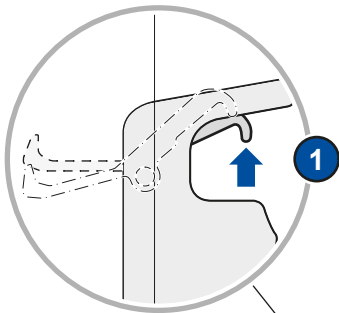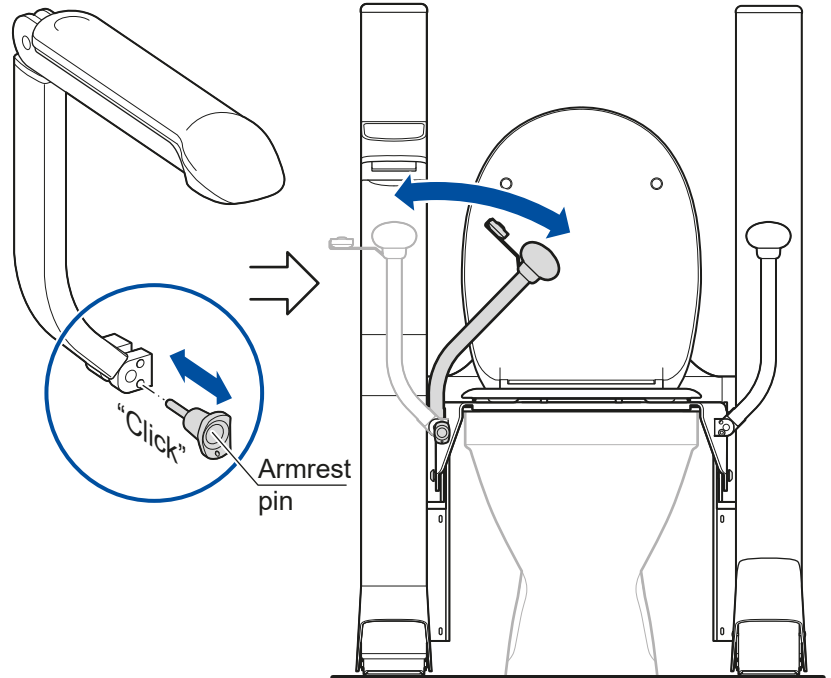Install the battery in reverse order (3, 2, 1). Tilt the armrest back and push the pin in completely (click/magnet).

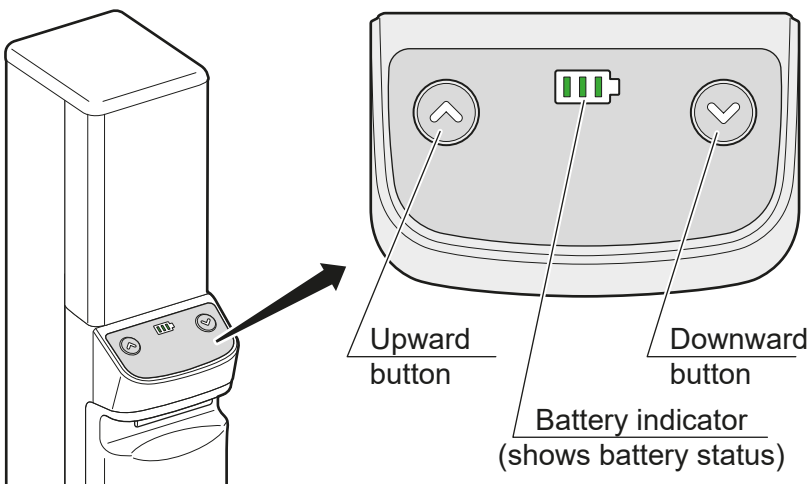
Remote control/control box/charge battery

Battery power/charging!

When the battery power starts to decrease, a warning signal will sound and the battery indicator will turn orange. Before the battery stops working, you can only drive the **LIFT Easy** downwards. The battery will charge automatically if the power cable is connected and the battery is in place. The power cable can be connected continuously for continuous charging.

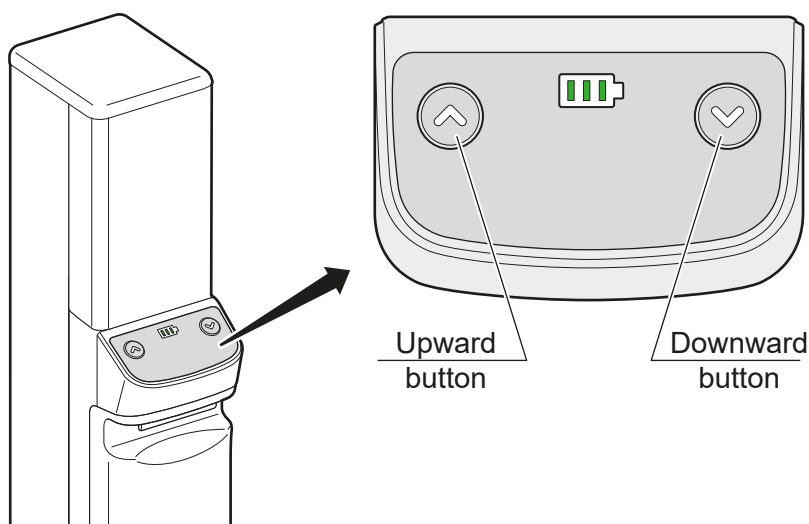Calibration

NOTE!

If **LIFT Easy** does not go all the way up/down, or for some other reason needs to be calibrated: Press and hold both buttons (up and down) on the control box for 10 seconds. The lift will now move slowly downwards. Continue to hold both buttons until both motors stop (this may take up to 2 minutes). **LIFT Easy** is now calibrated and can be used as normal.

Before use

Check that **LIFT Easy** is firmly attached to the toilet and that it is stable according to the installation instructions before use.

Cleaning and maintenance

If necessary, clean the product with a mild, environmentally friendly detergent and water. Disinfect if necessary.

Test run **LIFT Easy** at least once a month. Check that there is no visible damage to the mains cable or other components. If damage is suspected, the product should not be used.

If the mains cable is damaged, it must be replaced with an original spare part by the manufacturer, its retailer/service workshop or other qualified personnel, in order to avoid danger. Service must not be carried out while the product is in use. No accessories other than those specified and supplied by the manufacturer may be used.

Reconditioning

To extend the life of the product, there are a large number of components that are replaceable and should be replaced if they become damaged or worn. See the website for a complete list and part numbers. Check that there is no visible damage. Clean **LIFT Easy** as described above. Toilet seat, remote control and grip parts on the armrests can be replaced if damaged or worn. Replace battery if necessary.

Materials, environment and recycling

LIFT Easy is developed with high-quality components for long-term reuse and easy maintenance. The product is easily disassembled and can be recycled into different material fractions. The packaging is made of recycled material and the cardboard is made of recycled fiber. The columns are made of painted aluminum. The feet are made of painted steel sheet. The fixed and movable bridge is made of painted steel. The battery module contains electronics and is recycled as electronic scrap. The battery contains lead and acid and is recycled separately. Metal parts are recycled as metal.

Accessories

LIFT Easy is designed to be used with **Bidette R3** or **Bidette Easy**.

The following accessories can be used with the product:

<i>Designation</i>	<i>Item no.</i>	<i>Designation</i>	<i>Item no.</i>
Bidette R3	11 122	External charging station	11 792
Bidette Easy	17 000	Battery (extra)	11 794

Svancare reserves the right to make changes to the parts list. Current information is available on our website.

Parts list

Pos.	Nr.	Designation	Item number
1	1	Pillar with battery and control box	
2	1	Battery Lift Easy	11 794
3	1	Pillar	
4	1	Toilet seat (soft-closing with quick release)	11 904
5	1	Fixed bridge	
6	1	Movable bridge	
7	1	Cover	
8	1	Seat plate	
9	1	Armrest right	
10	1	Armrest left	
11	1	Armrest pin	11 905
12	1	Remote control, Lift Easy	11 777
13	1	Holder remote control, Lift Easy	11 467
14	1	Mains cable (C7-EU)	11 910
15	2	Nut (included in the mounting kit)	
16	2	Screw (included in the mounting kit)	
17	2	Grip part PUR Gray	15 357
18	4	Screw	
19	1	External charging station, Lift Easy	11 792
20	4	Screw	
22	2	bushing (included in the mounting kit)	
23	2	Washer (included in the mounting kit)	
24	1	Key (included in the mounting kit)	
25	1	Screw	11 907